Monteringsvejledning (DK)

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Mounting instruction

R82[®]

Mounting the fixed side supports and quick locks

Follow the instructions below to mount the fixed side supports (x:panda):






1. Place the two nuts (A) in the rails of each side support.
2. Place the desired side supports in the slot (B) behind the back cushion.
3. Fasten the side support using the two enclosed bolts (C).

Follow the instructions below to mount the fixed side supports (Panda Futura, active size 5):

1. Place the side supports in the slots.
2. Place the nuts, one by one, through the hole (G) in the back and into the slots in the side support.
3. Fasten from behind with enclosed bolts (H) and 5 mm Allen key.
4. Adjust the height and fasten.

Follow the instructions below to mount the quick locks:

1. Place the two nuts (D) in the rails of each side support.
2. Place the desired side supports in the slot (E) behind the back cushion.
3. Fasten the quick lock together with the side support using the two enclosed bolts (F).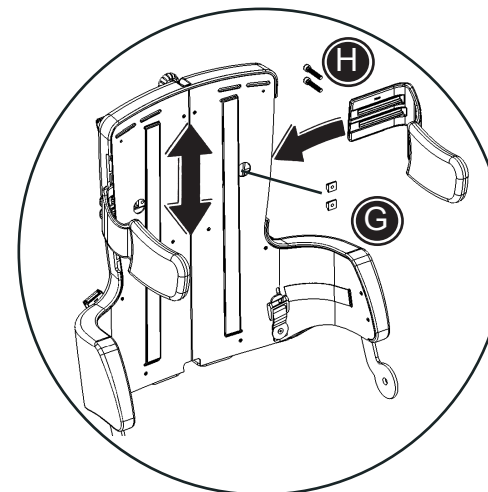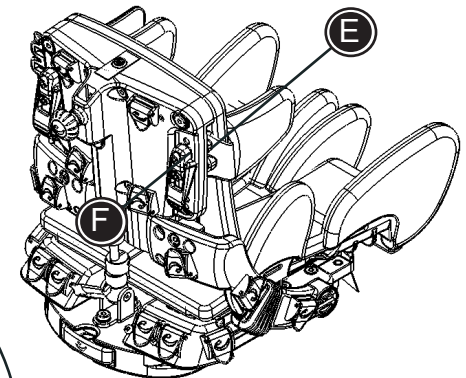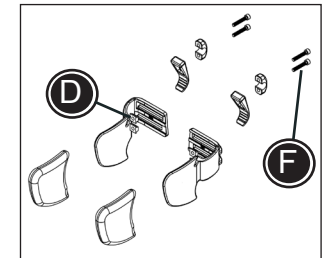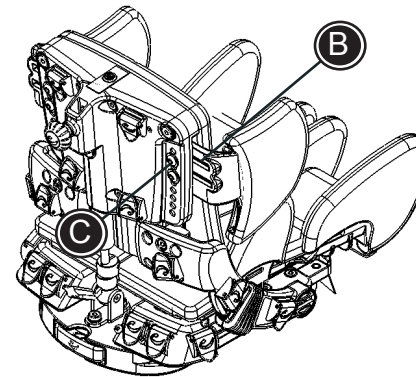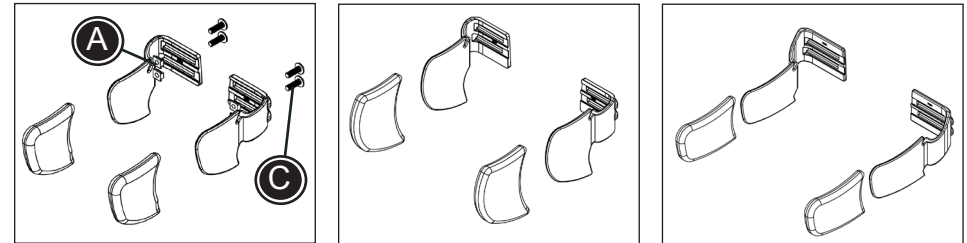

 Any adjustments and addition of accessories must be placed and fitted correctly and checked before every use.

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