Monteringsvejledning (DK)

Montering af hoftestøtter

Monteringsveiledning (N)

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Mounting instruction (GB)

Mounting the hip supports

Montageanleitung (DE)

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Montagehandleiding (NL)

De bekkensteunen monteren

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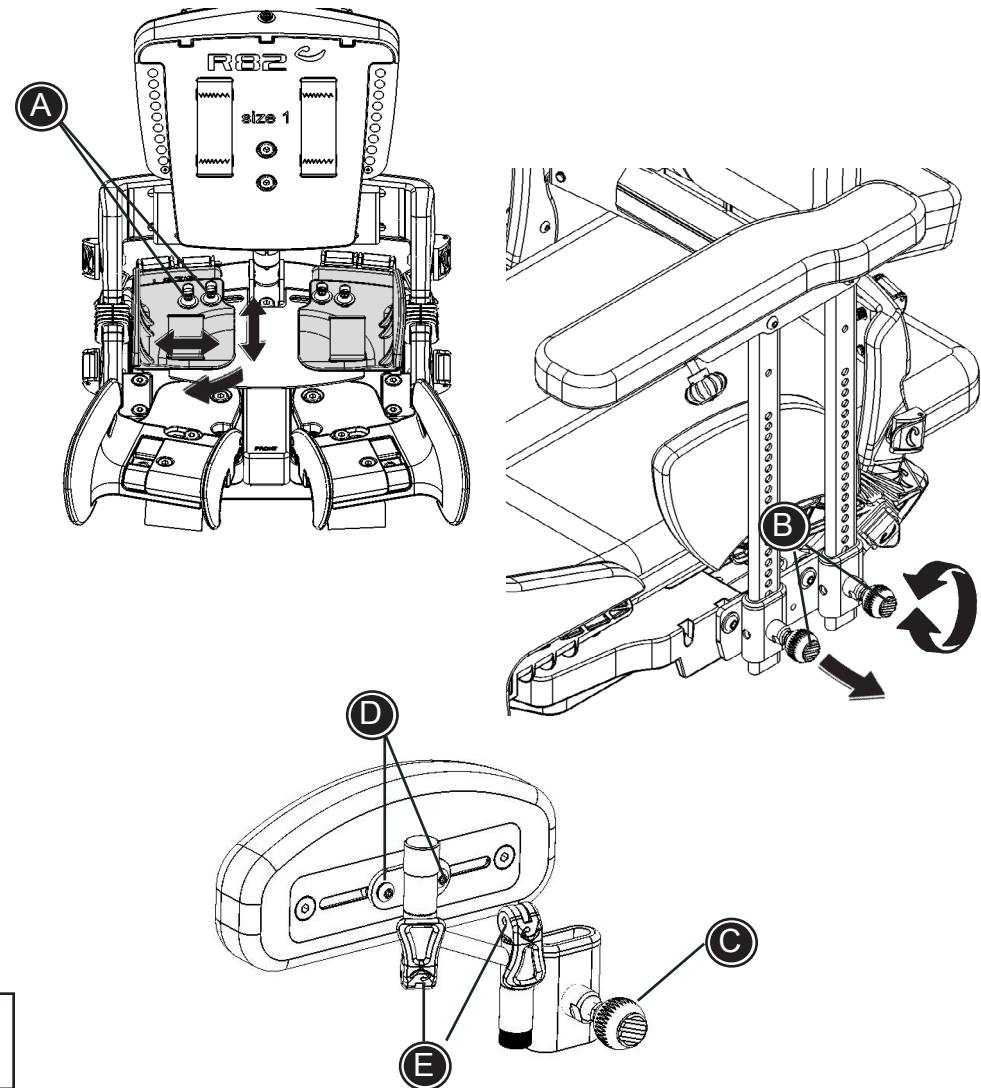
R82[®]

Mounting hip supports for x:panda 4

Follow the instructions below to mount the hip supports:

1. Dismount the standard hip supports by unscrewing (A) by the means of the Allen key that came with the chair. When the supports have been removed the screws can be remounted in the fittings for possible later use.
2. Dismount the arm rest bars from the fitting on the chair by loosening the handle/screw (B) by hand, and then the arm rest bar is removed.
3. Mount the hip support on one of the rods of the arm rest bar (of your own choice) and the handle/screw (C) is tightened by hand.
4. Mount the arm rest bars in the fitting on the chair again and tighten the handle/screw (B) by hand.
5. Use the enclosed 3 mm Allen key to loosen the screws (D) and adjust the depth in the slots and fasten in the required position.
6. The hip supports are multi-adjustable by moving the handles (E) up/down. The desired position is found and if necessary the fittings are tightened before the handles are moved up/down again.

Any adjustments and addition of accessories must be placed and fitted correctly and checked before every use.

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