Monteringsvejledning (DK)

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






Mounting instruction

Mounting transport fitting on the Stingray


Follow the below instruction to mount the transport fitting on the Stingray and prepare the pushchair to transportation in vehicles: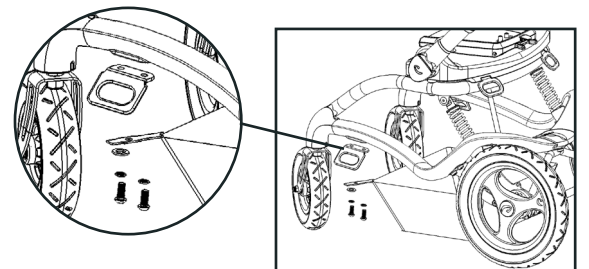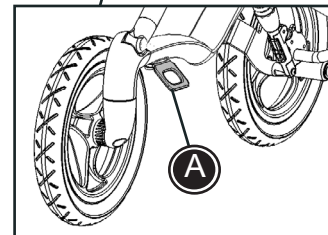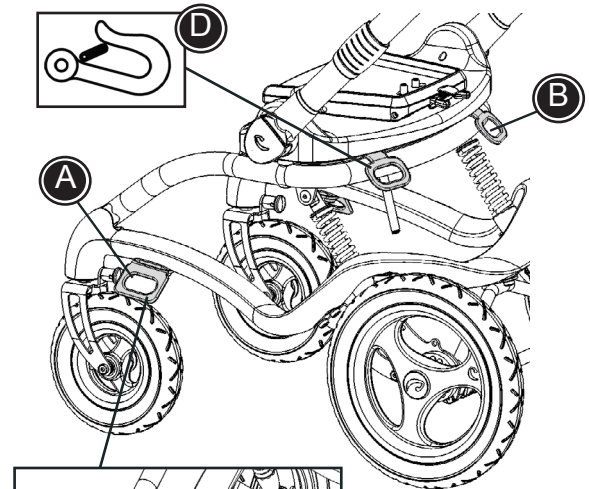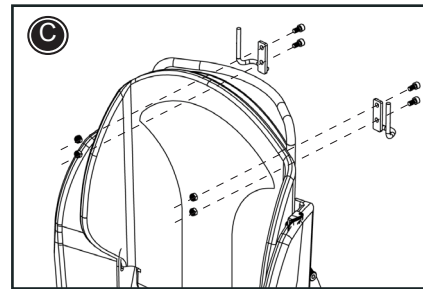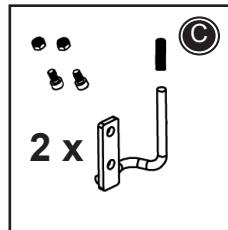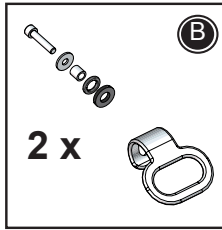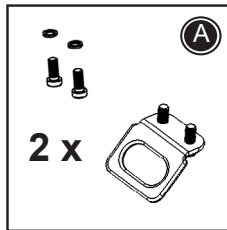

Secure the child in the pushchair:

Mount a hip-belt, approved according to ISO 10542-1 or SAE J2249 and secure the child in the pushchair.

The back MUST be in vertical position and the security locks MUST be locked.

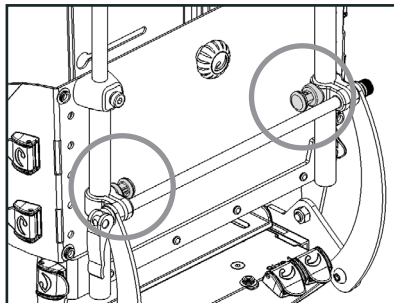Secure the Stingray in the vehicle:

Mount two fittings (A) on both sides of the pushchair. If using the shopping basket the same holes and bolts are used - ensure they are mounted in the correct order, follow the drawing below right.

Mount the two fittings (B) on the back of the pushchair. The transport fittings (A) and (B) must be equipped with a secure symbol (D).

The red seat belt guides (C) are mounted using the pre-drilled holes in the seat back as shown in drawing above. Ensure the black PVC caps are fitted.

Secure the pushchair to the vehicle, using a 4-point strap type tiedown system, approved according to ISO 10542-2 and SAE J2249.

Use the red seat belt guides to secure the vehicle's 3-point safety belt around the seat and user. Ensure all restraints are tightly secured, but comfortable for the user, and that belt webbing is not twisted.

Please find the Stingray manual on www.R82.com

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