Monteringsvejledning (DK)

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Monteringsveiledning (N)

Montering av R82 regncape med hette

Mounting instruction (GB)

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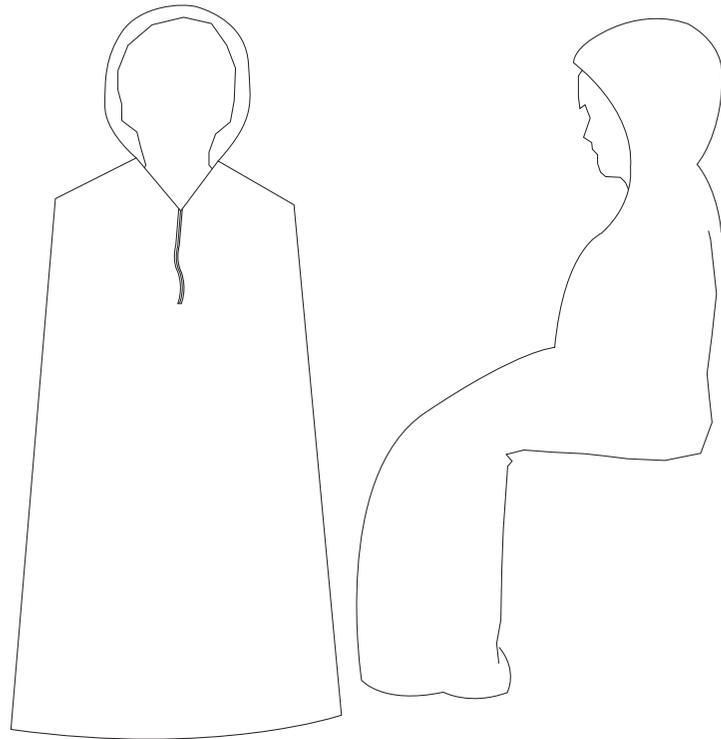
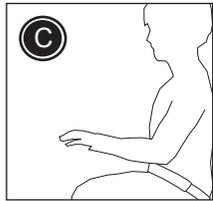
Producer
R82 A/S
Parallevej 3
DK-8751 Gedved

Mounting instruction

How to mount the R82 Raincape with hood

Recommended for: x:panda, Panda Futura, Panther, Tiger 2000, Serval, Cougar

The raincape goes over the back of the seat. Around the users body and under the footrest. In this way it also covers the chair. Any doubt on mounting, please contact a local therapist or authorized R82 personnel.

 To obtain the best seating position and to prevent the user from sliding down and out of the chair, we recommend always to mount a hip belt (C) together with the rain cape.

 Never leave the user unattended. Ensure permanent supervision by an adult. Take care that all fixations and adjustments are placed and fitted correctly. Incorrect fitting may cause serious injury. Check before every use.

 All buckles must be checked and cleaned on a regular basis. To obtain optimal safety, always keep the buckles cleaned.

 Carefully read the care label on the raincape to maintain and wash correctly.

 The latest version of the mounting instruction is always available online: R82.com

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