Monteringsvejledning (DK)

Montering af R82 Y-sele – polstret

Monteringsanvisning (SV)

Montera R82 Y-bälte – polstrat

Monteringsveiledning (N)

Montering av R82 Y-sele – polstret

Mounting instruction (GB)

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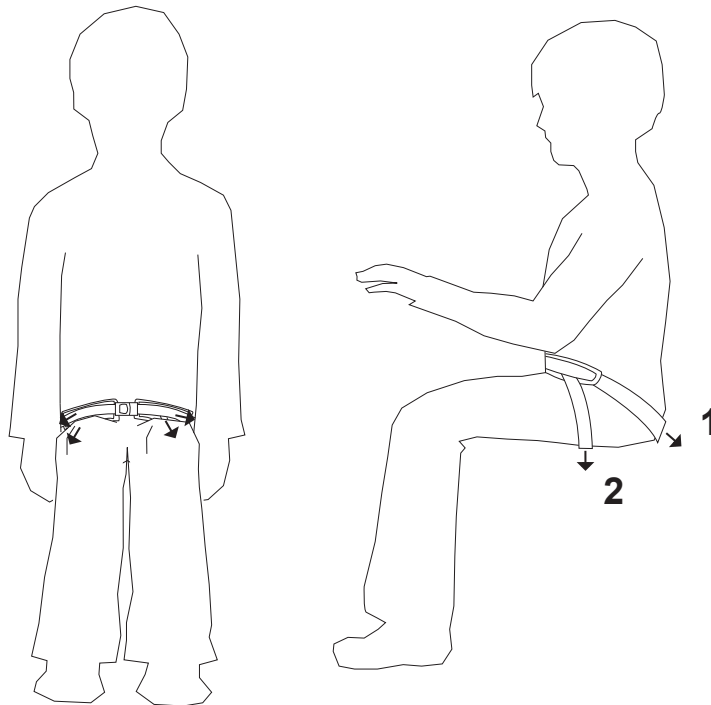
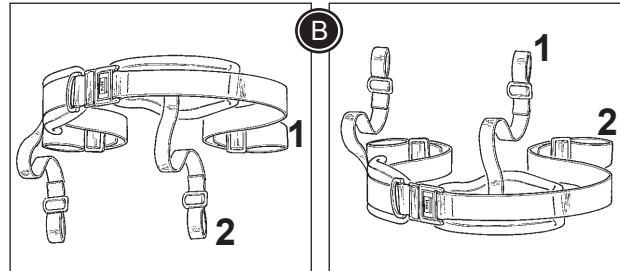
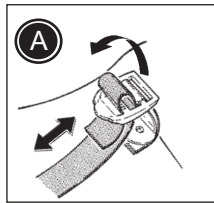
Producer
R82 A/S
Parallevej 3
DK-8751 Gedved
www.R82.com

Mounting instruction

How to mount the R82 Y-belt - upholstered

Recommended for: x:panda, Panda Futura, Panther, Tiger 2000, Serval, Wombat, Cheetah, Cougar, Winther Reha

Use the fittings (A) on the chair to mount the Y-belt. If no fittings are mounted, please use the enclosed buckles to mount the Y-belt. Any doubt on mounting, please contact a local therapist or authorized R82 personnel.

For the best fit you can mount the R82 Y-belt in two ways (B).

- * Mount the upper straps (1) at about 45° into the back of the seat.
- * Mount the lower straps (2) over the thighs.

 Always tighten all straps securely to obtain optimal safety, support and comfort. The Y-belt is fitted correctly when two fingers can be put between the harness and the child's body. Check before every use.

 Never leave the user unattended. Ensure permanent supervision by an adult. Take care that all fixations and adjustments are placed and fitted correctly. Incorrect fitting may cause serious injury. Check before every use.

 All buckles must be checked and cleaned on a regular basis. To obtain optimal safety, always keep the buckles cleaned.

 Carefully read the care label on the harness to maintain and wash the Y-belt correctly.

 The latest version of the mounting instruction is always available online: www.R82.com

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