Monteringsvejledning (DK)

Montering af R82 hoftesele - polstret

Monteringsanvisning (SV)

Montera R82 höftbälte – polstrat

Monteringsveiledning (N)

Montering av R82 hoftesele - polstret

Mounting instruction (GB)

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

Producer
R82 A/S
Parallevej 3
DK-8751 Gedved

Mounting instruction


How to mount the R82 Hip belt - upholstered

Recommended for: x:panda, Panda Futura, Panther, Tiger 2000, Serval, Wombat, Cheetah, Cougar, Winther Reha

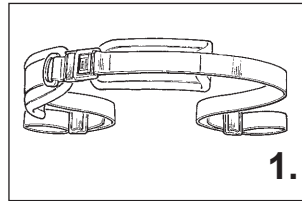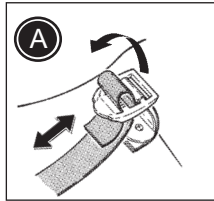Use the fittings (A) on the chair to mount the hip belt. If no fittings are mounted, please use the enclosed belt fittings to mount the hip belt. Any doubt on mounting, please contact a local therapist or authorized R82 personnel.

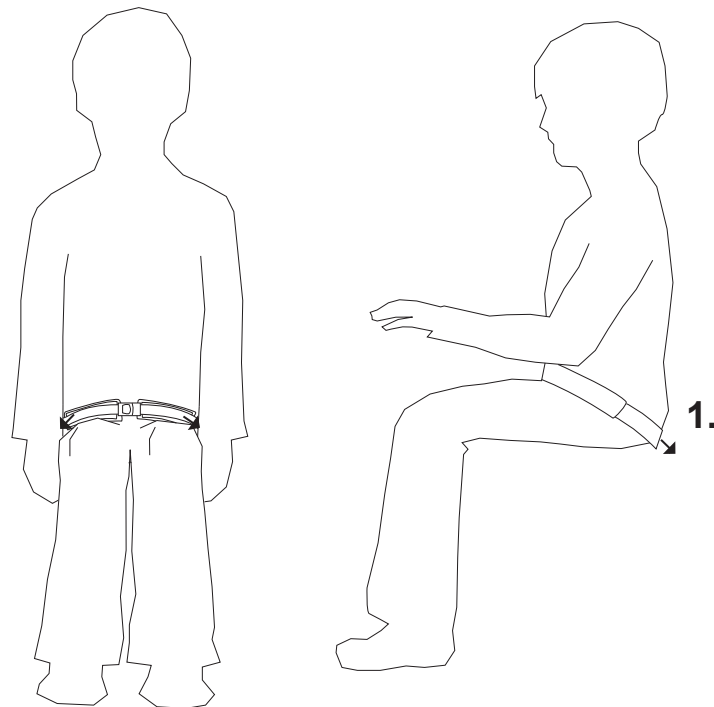* Mount the lower straps (1) in the seat.

 Always tighten all straps securely to obtain optimal safety, support and comfort. The hip belt is fitted correctly when two fingers can be put between the belt and the child's body. Check before every use.

 Never leave the user unattended. Ensure permanent supervision by an adult. Take care that all fixations and adjustments are placed and fitted correctly. Incorrect fitting may cause serious injury. Check before every use.

 All buckles must be checked and cleaned on a regular basis. To obtain optimal safety, always keep the buckles cleaned.

 Carefully read the care label on the harness to maintain and wash the hip belt correctly.

 The latest version of the mounting instruction is always available online: R82.com

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