## Monteringsvejledning (DK)

Montering af adapter (adaptor 8911610-98)

## Monteringsanvisning (SE)

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## Monteringsveiledning (N)

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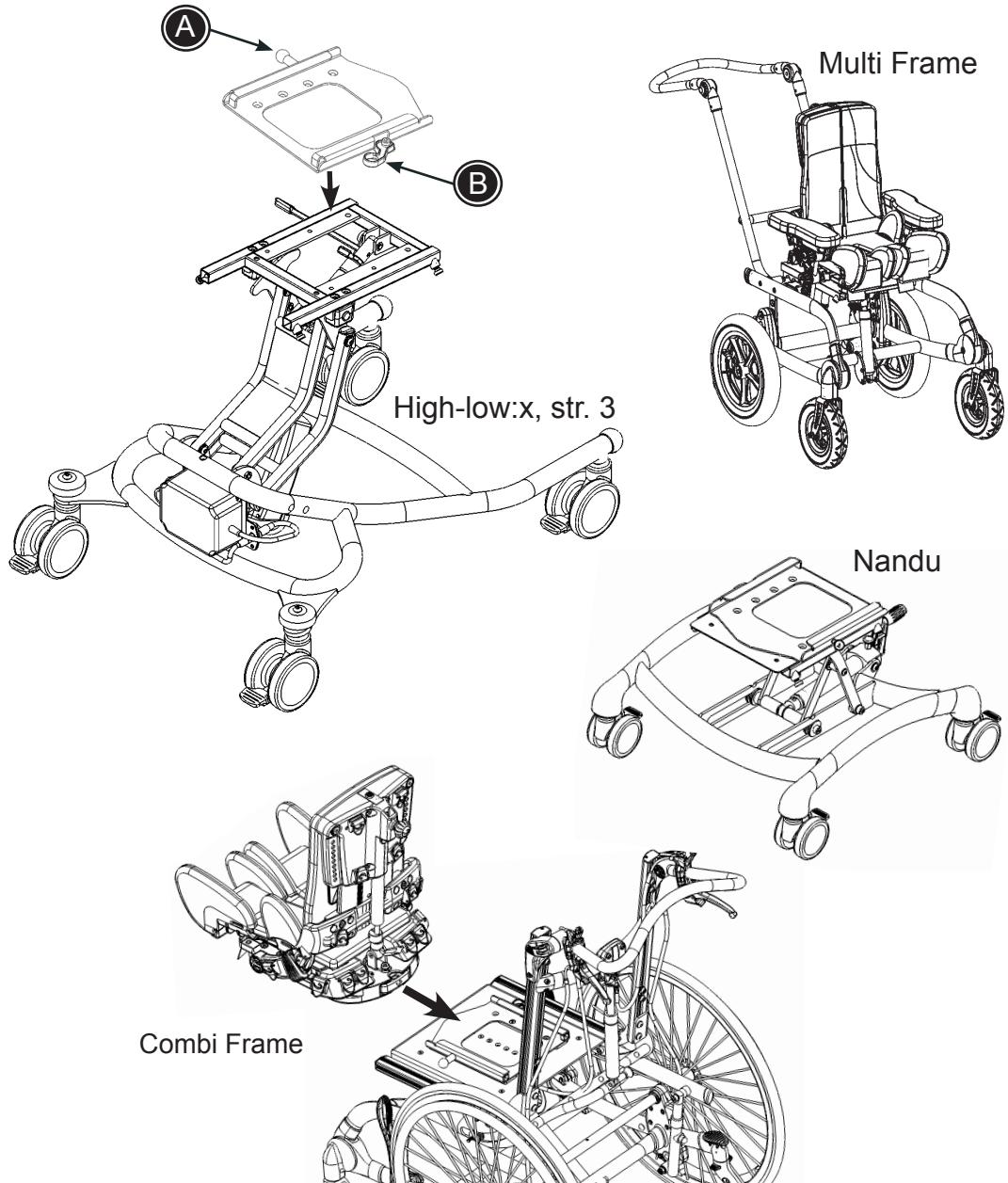
# Mounting instruction

Mounting an interface and seat (interface 8911610-98)

Follow the instruction below to mount an interface for **x:panda and Panda Futura on Combi Frame, High-low:x sz 3, Multi Frame or Nandu frame:**

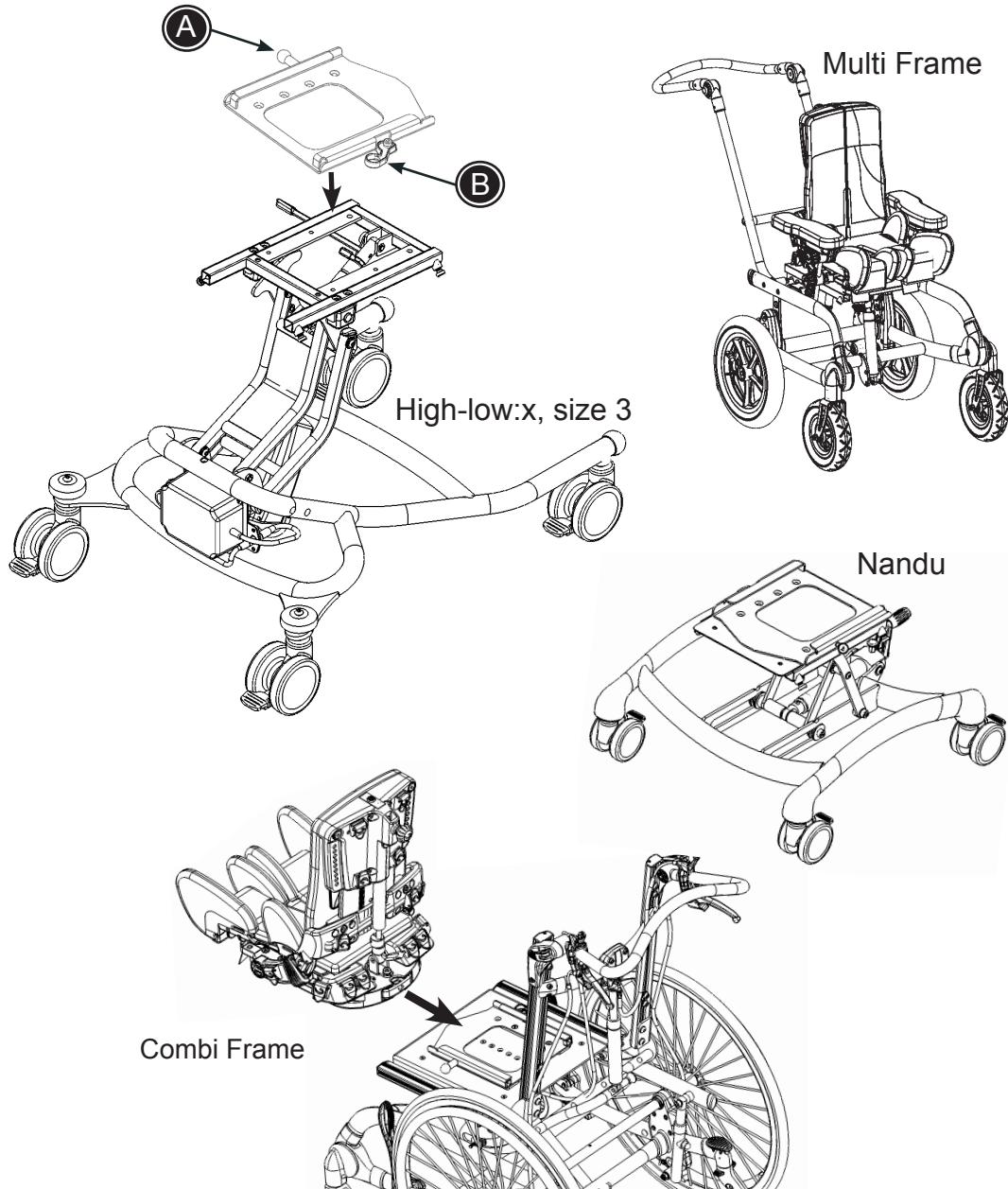
1. Place the interface in such way that you obtain the correct centre of gravity for the seat. The correct centre of gravity depends on the type of seat and seat depth. Therefore the interface can be moved backward and forward, depending on the desired placement.
2. Mount the enclosed bolts from the top by fastening with a nut beneath the seat base. Fasten all screws securely using a 4 mm Allen key and a 10 mm spanner.
3. The seat is slid into in the adapter and should be pushed back until it "clicks". Then pull out the green knob (A) and push the seat back until the green knob clicks into the second safety hole. Lock with the red handle (B) on the opposite side by levering the red handle towards the rear of the frame.

**!** Do not use the first hole the seat „clicks” into. This is just the „safety hole” which will „catch” the seat if the knob is accidentally released from its correct hole.

**!** Any adjustments and addition of accessories must be placed and fitted correctly and checked before every use.

**!** When mounting an interface on the Nandu: Use the triangular holes.

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