


Mounting instruction (GB)

Nandu seat on High-low frame

Mounting instruction

R82[®]

Nandu seat on High-low frame


 Mount the anti tilt restrictor on the High-low frame before use. The restrictor can be found in the tool bag which comes with the Nandu seat.

To mount the restrictor:

Click the anti tilt restrictor onto the gas spring as shown (A). Ensure that the restrictor is fitted before mounting the seat.

To mount the seat:

Place the seat in the adapter (B) and push it all the way back, until it "clicks" into place. Lock the red handle (C)

 Do not use the first hole it „clicks” into. This is just the "safety hole" which will "catch" the seat if the knob accidentally should pop out of its right hole.

To dismantle the seat:

Unlock the red handle (C). Pull out the knob (B), pull the seat forward, and lift at the same time.

Angling the seat:

Angle the seat by lightly pressing down the handle (E) and pushing against the seat back or the seat front for anterior tilt..

 The seat has upto 10° anterior and posterior tilt.

