## **Monteringsvejledning (DK)**

Montering af R82 ankelseler

## **Monteringsanvisning (SV)**

Montera R82 ankelselar

## **Monteringsveiledning (N)**

Montering av R82 ankelseler

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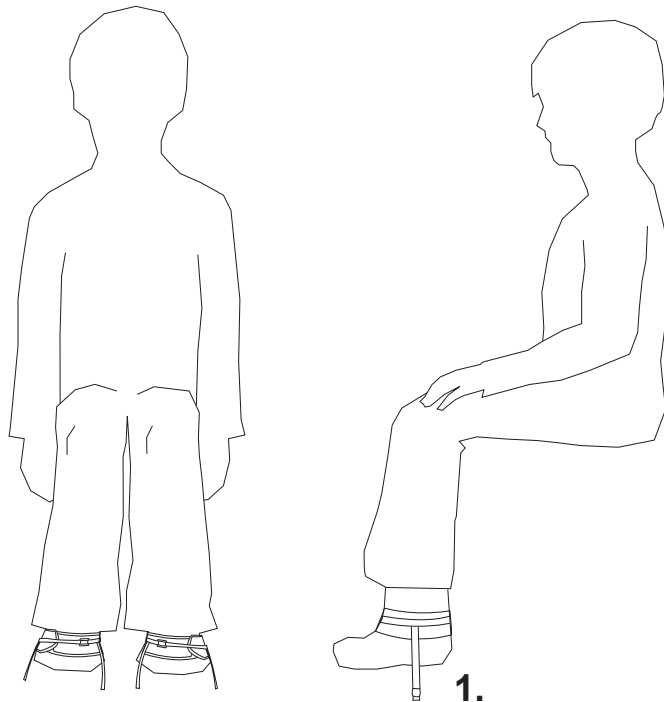
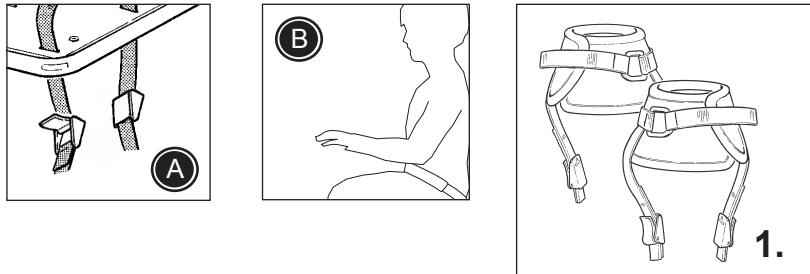
R82<sup>®</sup>

## How to mount the R82 Ankle straps

Mount the fix locks as shown on picture (A). Any doubt on mounting, please contact a local therapist or authorized R82 personnel.

 Always tighten all straps securely to obtain optimal safety, support and comfort. The ankle straps are fitted correctly when two fingers can be put between the straps and the child's foot. Check before every use.

 To obtain the best seating position and to prevent the user from sliding down and out of the chair, we recommend always to mount a hip belt (B) together with the ankle straps.

 Never leave the user unattended. Ensure permanent supervision by an adult. Take care that all fixations and adjustments are placed and fitted correctly. Incorrect fitting may cause serious injury. Check before every use.

 All buckles must be checked and cleaned on a regular basis. To obtain optimal safety, always keep the buckles cleaned.

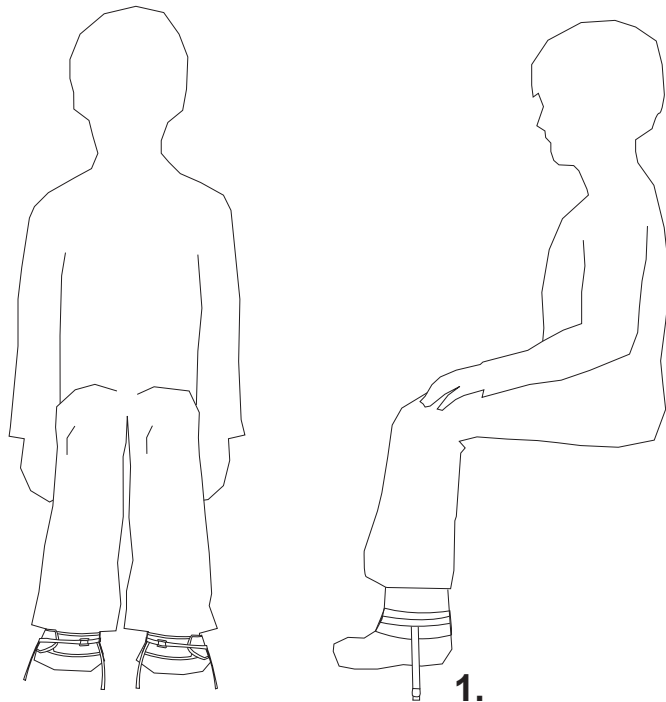
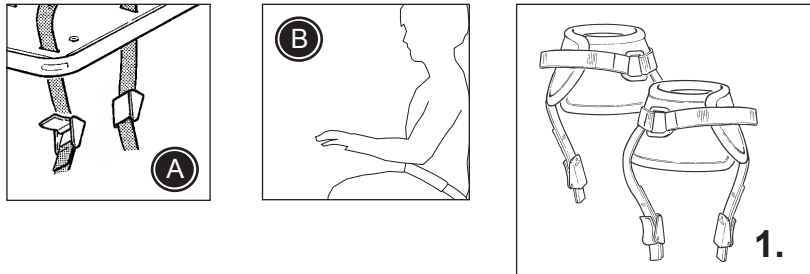
 Carefully read the care label on the harness to maintain and wash the ankle straps correctly.

 The latest version of the mounting instruction is always available online: [R82.com](http://R82.com)

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R82 A/S