58-73 cm (22 3/4"-28 3/4")

<100 kg (220.4 lb)


Combi Frame:x User Guide

EN 12183

2018-02 - rev.06

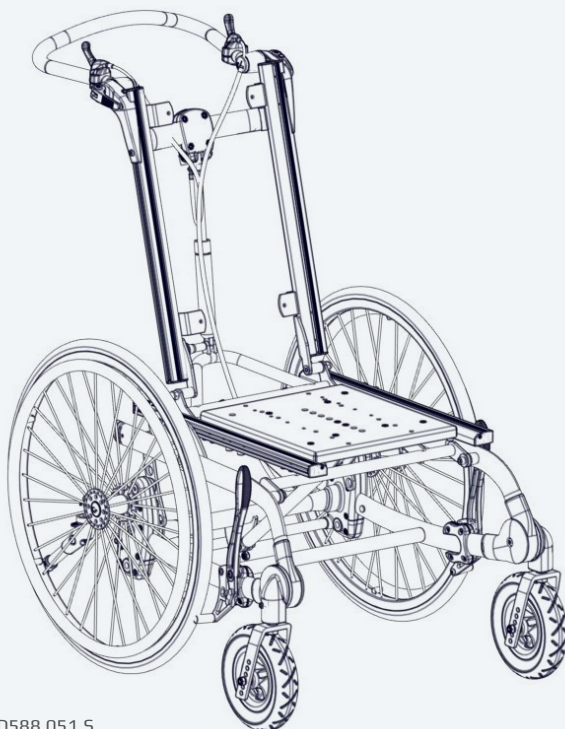	3		21
	4 - 5	mm (inch)	22
	6 - 7		23 - 24
	8	ENGLISH DEUTSCH	25 - 28 29 - 33
	9	NEDERLANDS DANSK	34 - 38 39 - 43
	10 - 11	NORSK SVENSKA	44 - 48 49 - 53
	12	SUOMI ESPAÑOL	54 - 58 59 - 63
	13	FRANÇAIS ITALIANO	64 - 68 69 - 73
	14	PORTUGUÊS PORTUGUÊS DO BRASIL	74 - 78 79 - 83
	15 - 16	РОССИЯ POLSKI	84 - 88 89 - 93
	17	ČESKY 中國	94 - 98 99 - 102
	18	日本 ΕΛΛΗΝΙΚΗ	103 - 106 107 - 111
	19		
	20		
	21		


U.S. Patent Des. D588,051 S
Norwegian Design Reg. No. 80196

Combi Frame:x


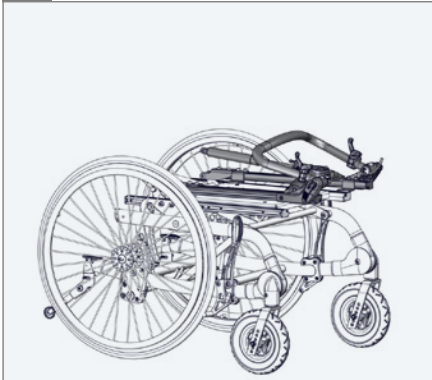
1.

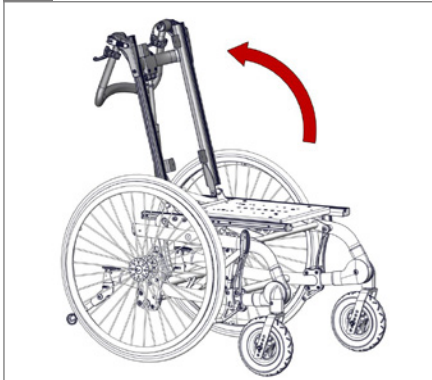
2.

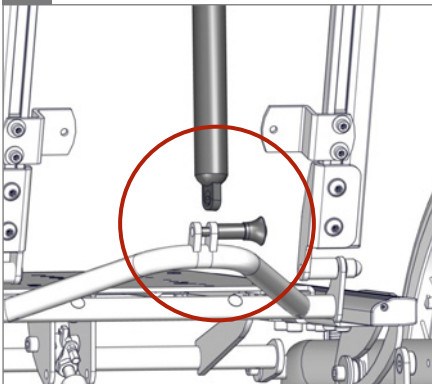
3.

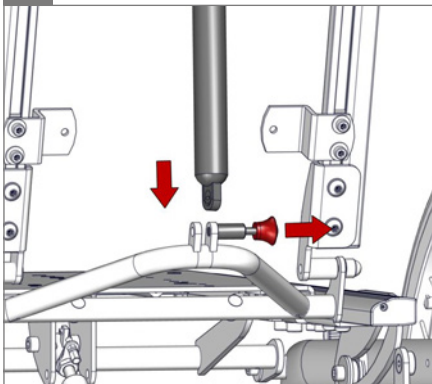
4.

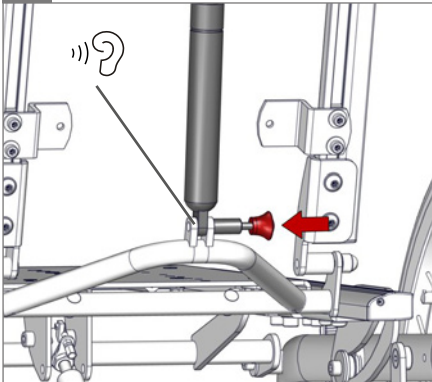5.

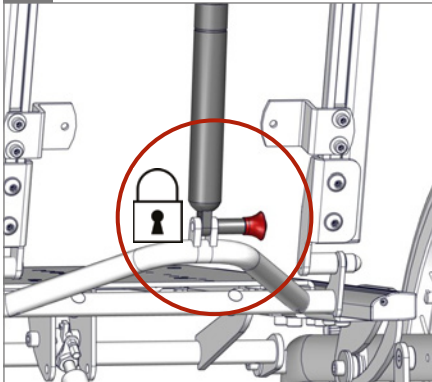6.



7.

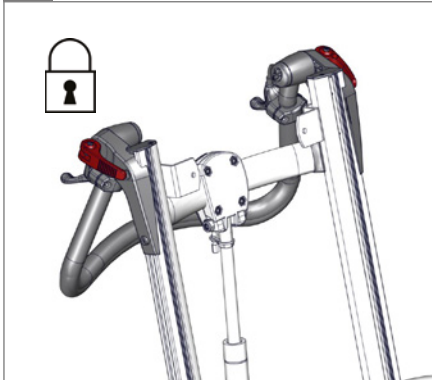
8.

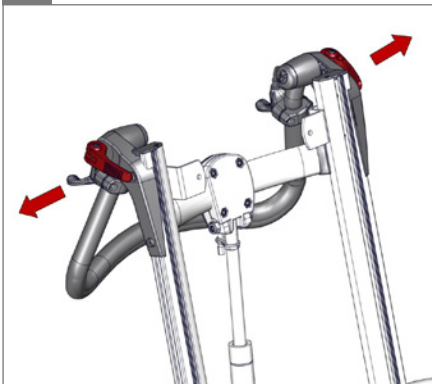
9.

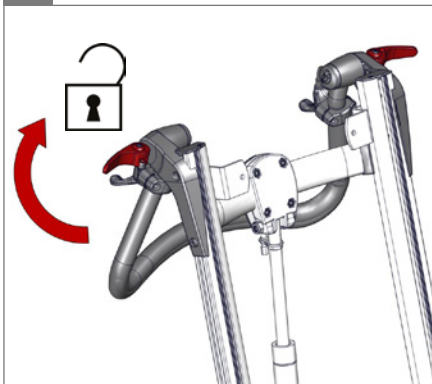
10.

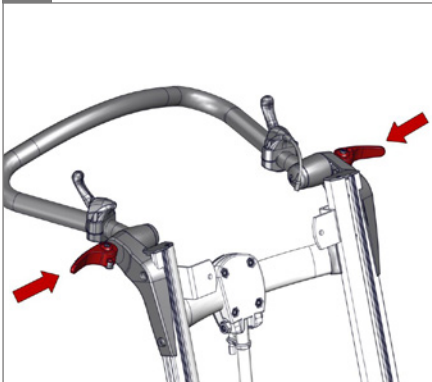11.

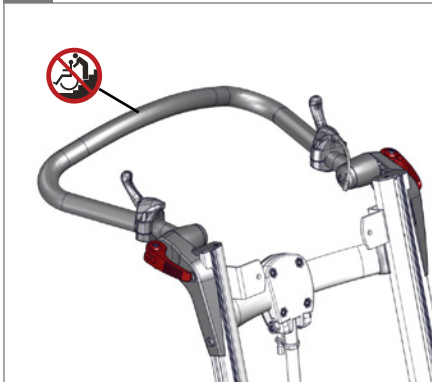
12.


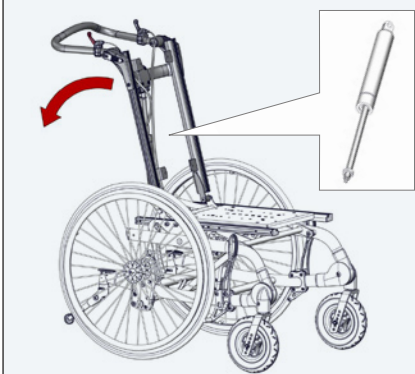
1.

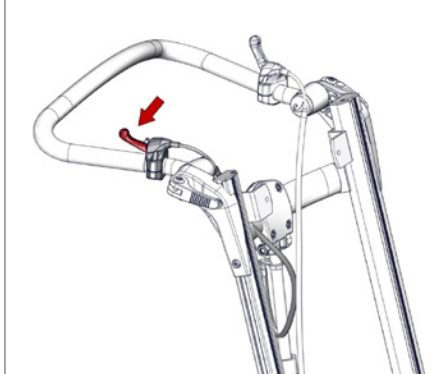
2.

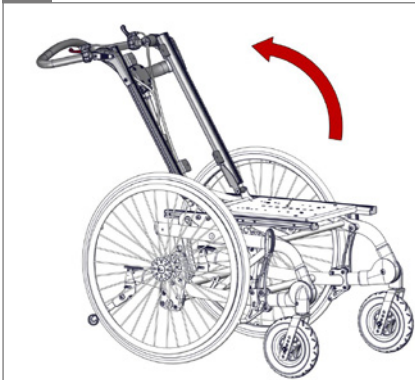3.

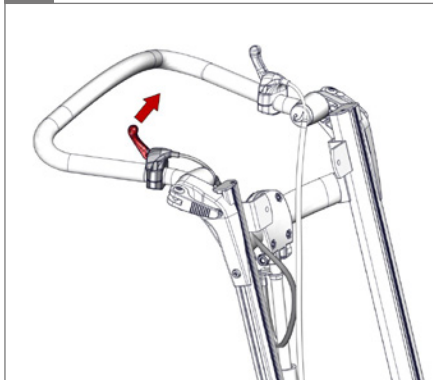4.


5.


6.

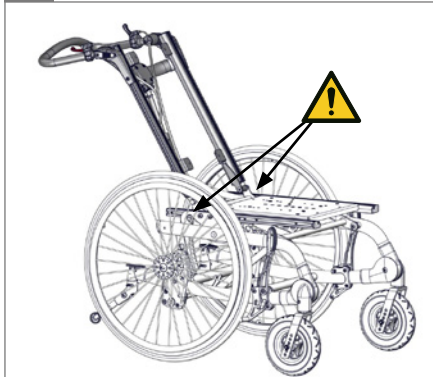01

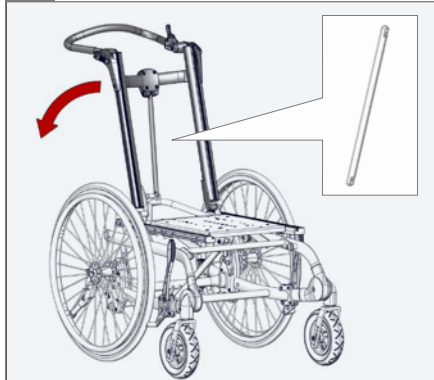
1.


2.

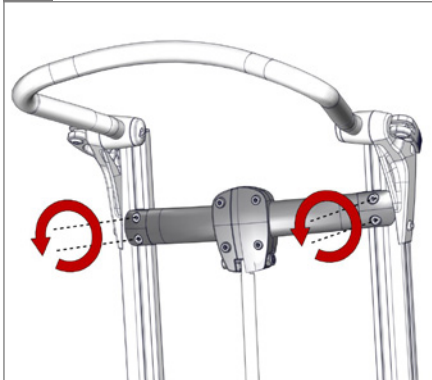⬡ 5 mm

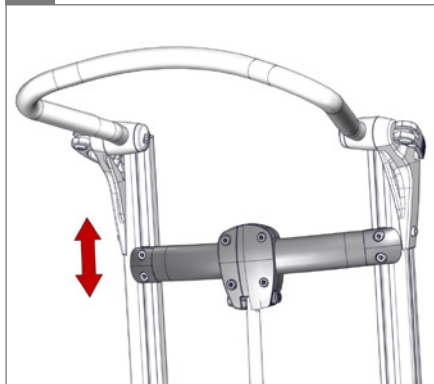
3.

4.

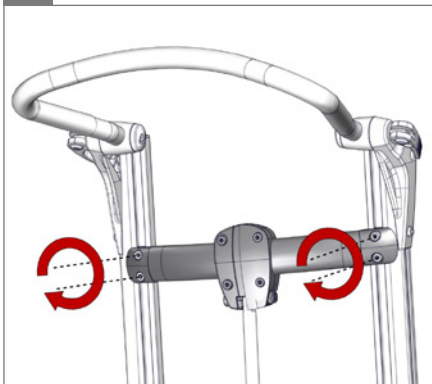⬡ 5 mm - 2,5 Nm

5.

6.

01 ⚠

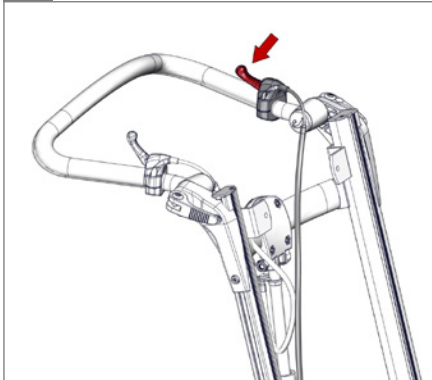
1.


2.

3.

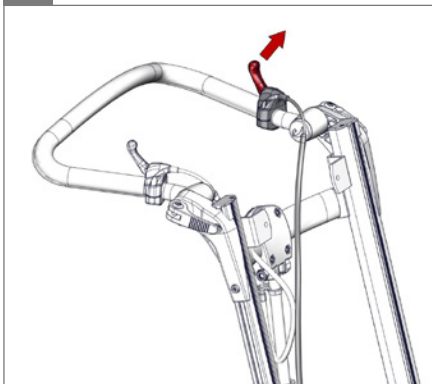4.


5.

01

6.

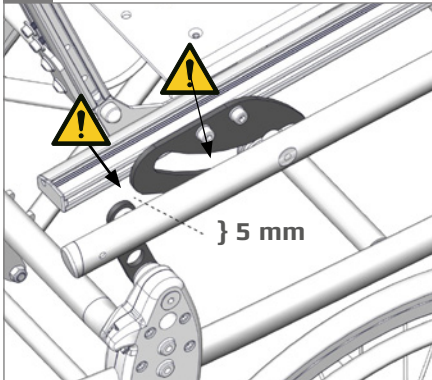01


1.

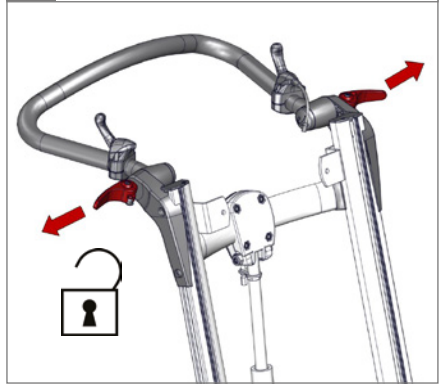2.


3.

01

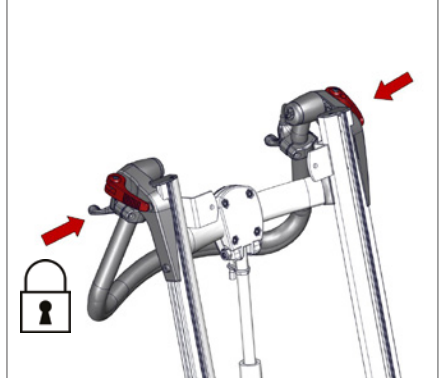
4.


5.

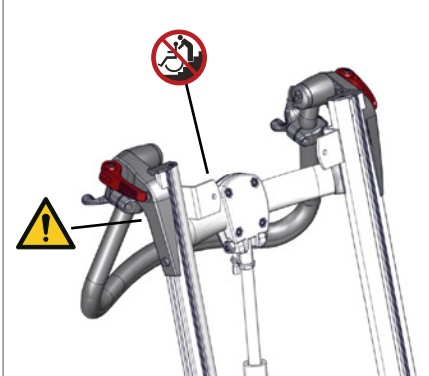01

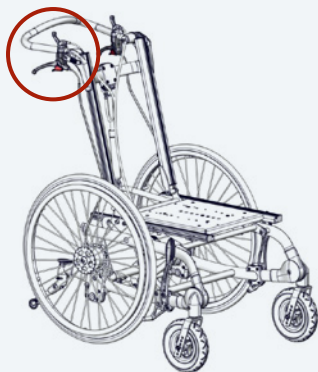
1.

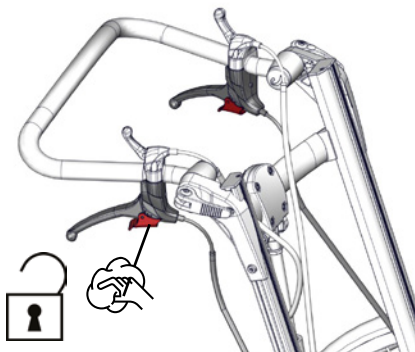
2.


3.

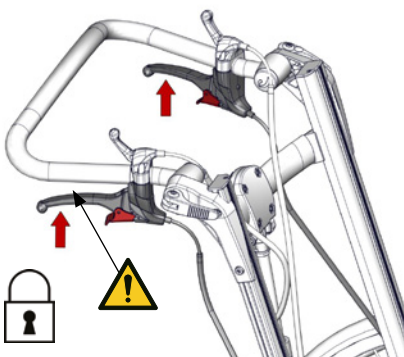
01

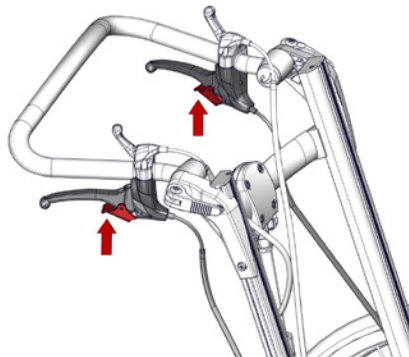4.

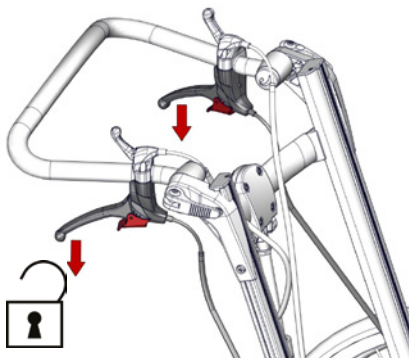
5.


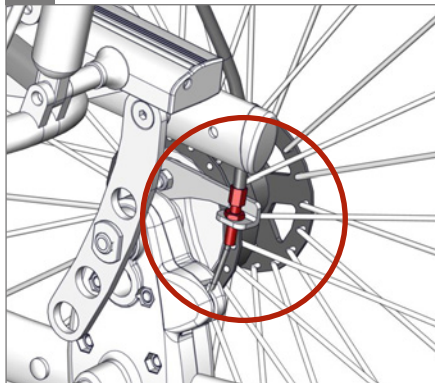
7.

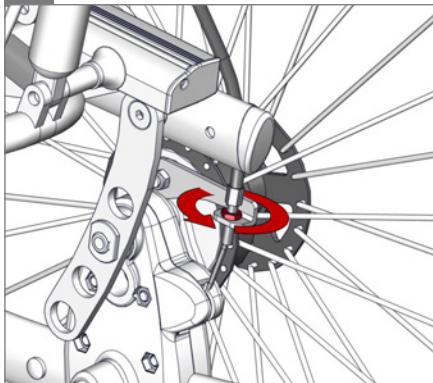
8.

 8 mm


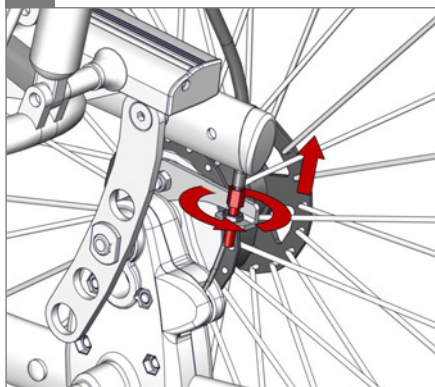
9.

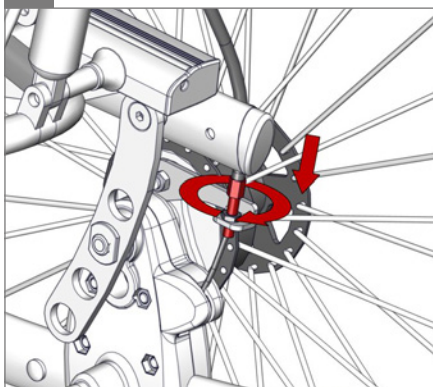10.

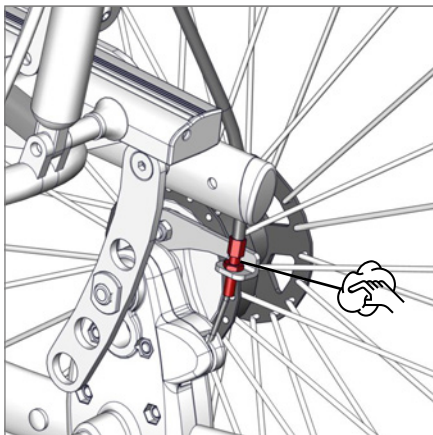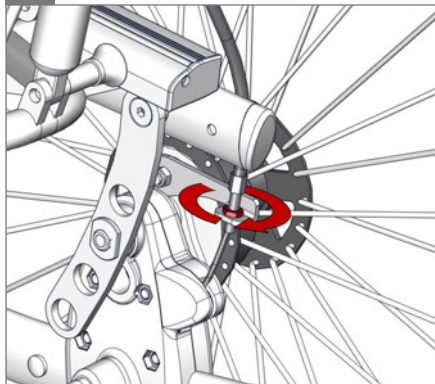
11.

 8 mm


1.

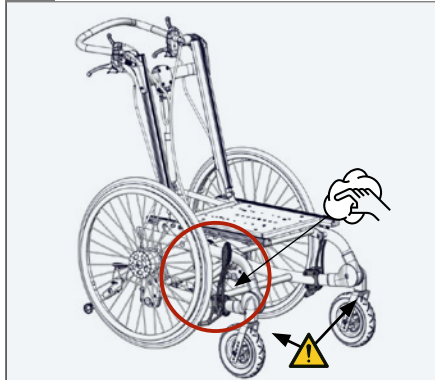01

2.

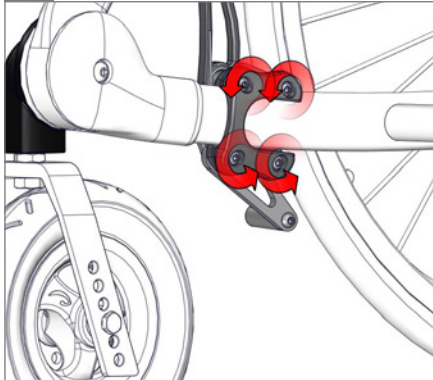
5 mm - 2,5 Nm


3.

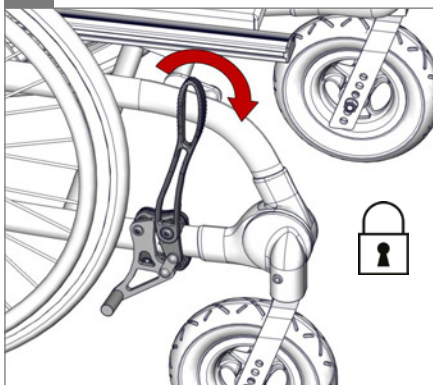
4.

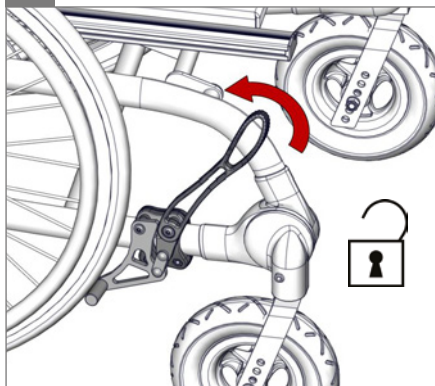5.

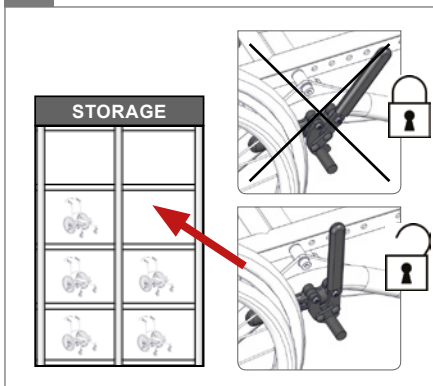
6.


1.

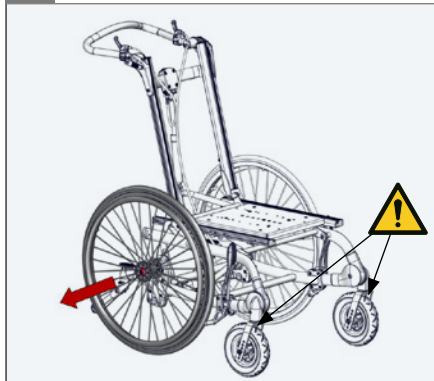
01

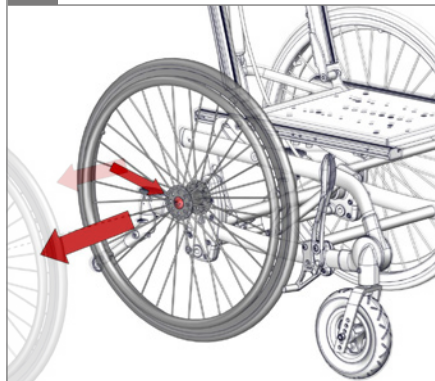
2.

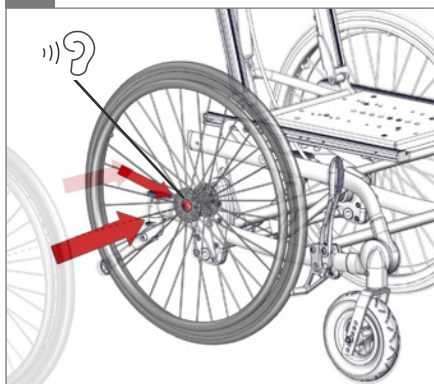3.




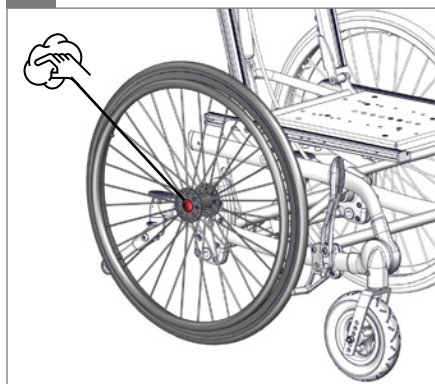
4.


	12 1/2"	20"	22"	24"
1	-	✓	✓	✓
2	✓	-	-	-
Handrim diameter		445 mm (17 1/2")	480 mm (19")	530 mm (20 3/4")

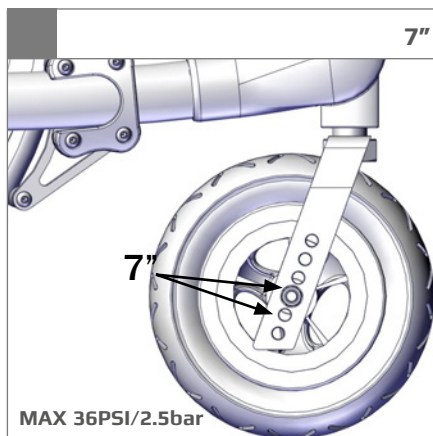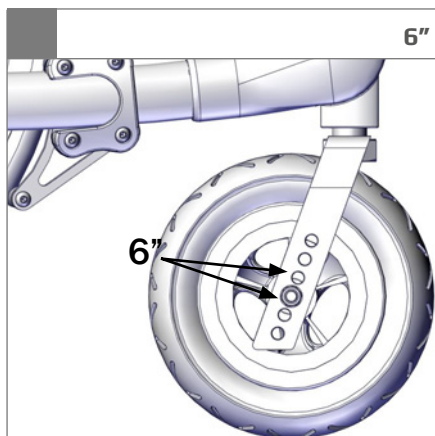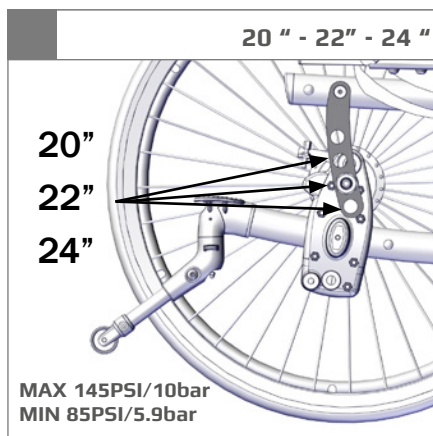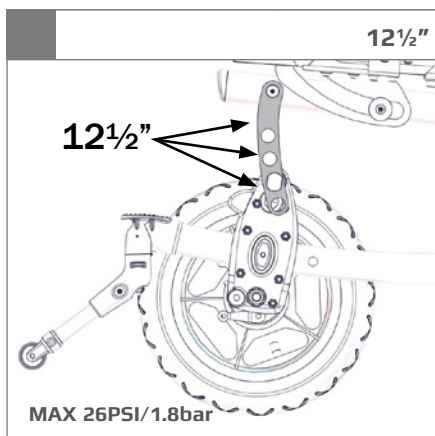
1.

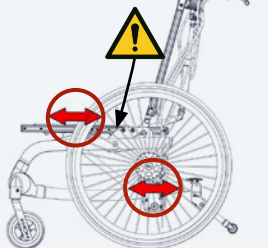02


2.

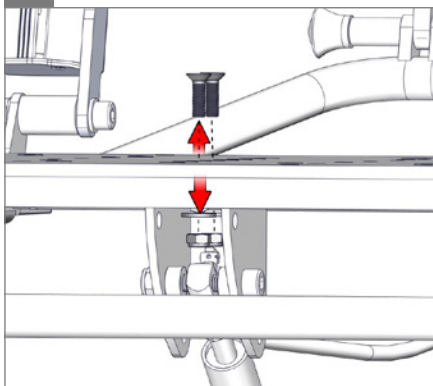⌀ 4 mm & 10 mm

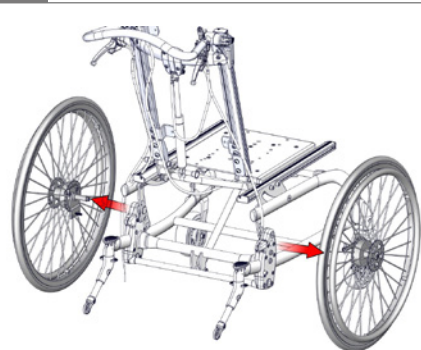
3.


4.

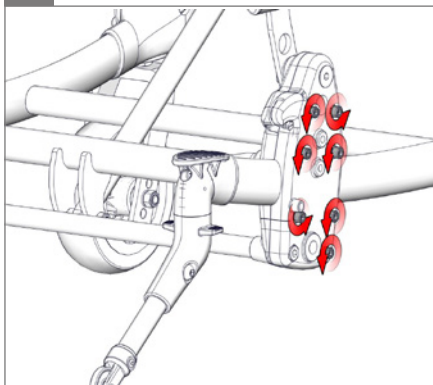
⌀ 5 mm

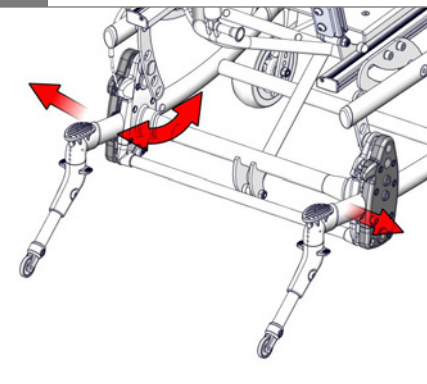5.

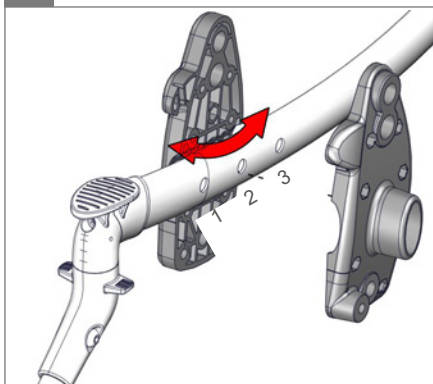
6.


7.

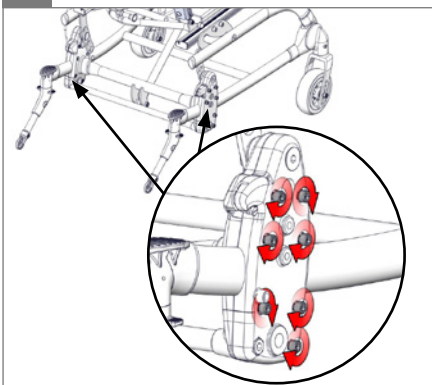
⬡ 5 mm

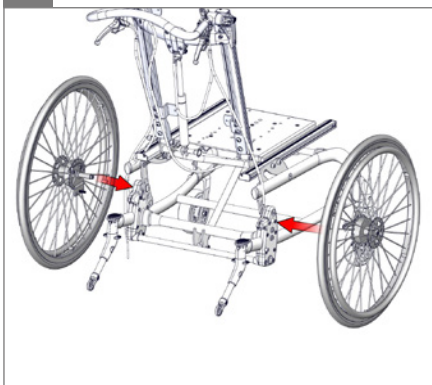8.

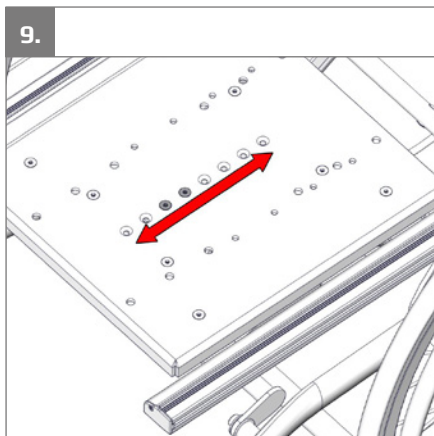
9.


10.

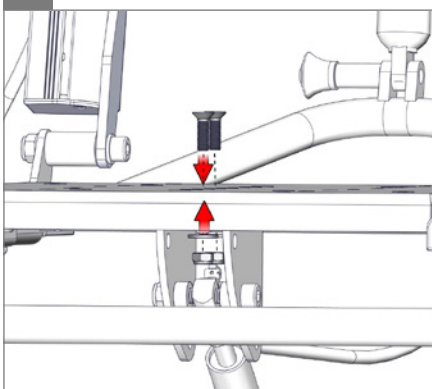⬡ 4 mm & 🔑 10 mm

11.

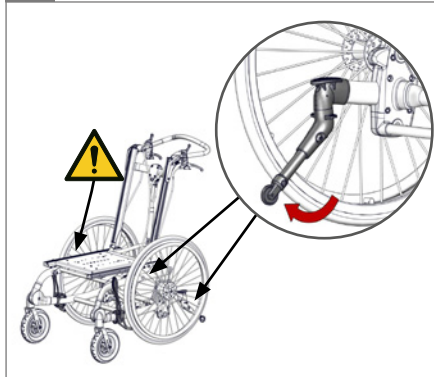02 ⚠



1.

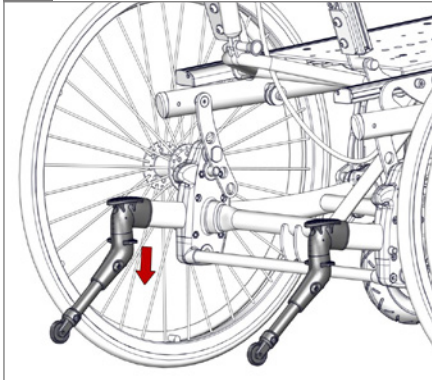2.

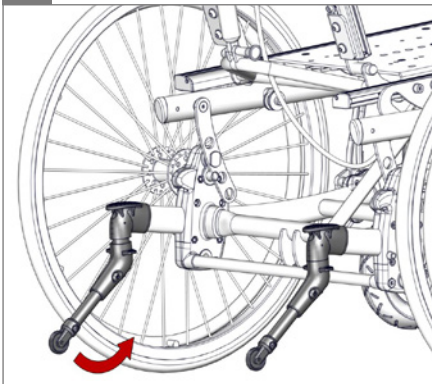
3.


4.

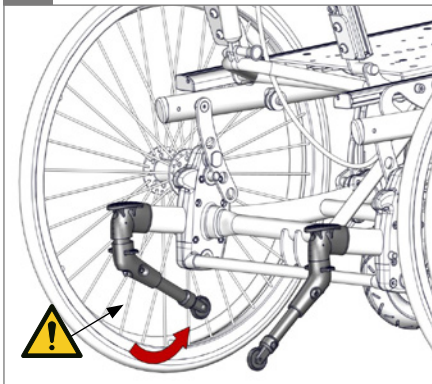
01

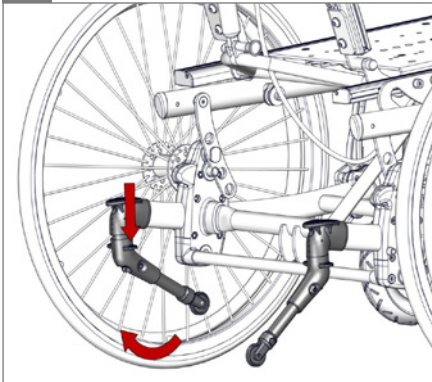5.

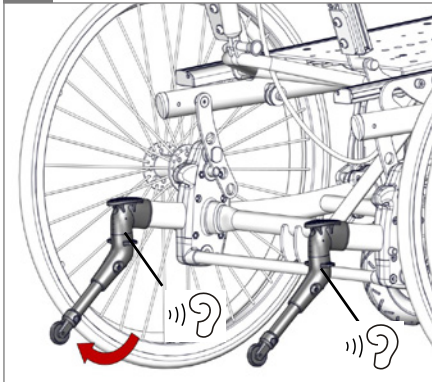
6.


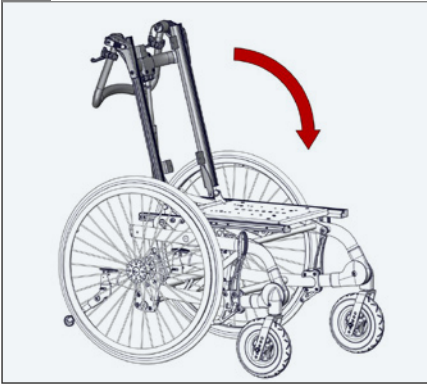
1.

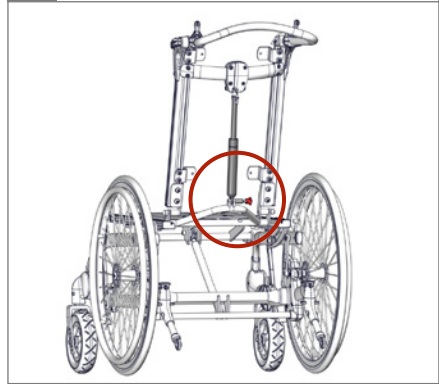
2.

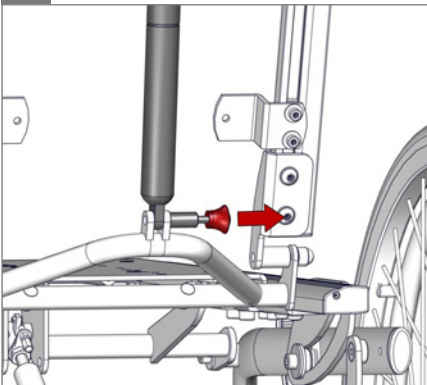3.

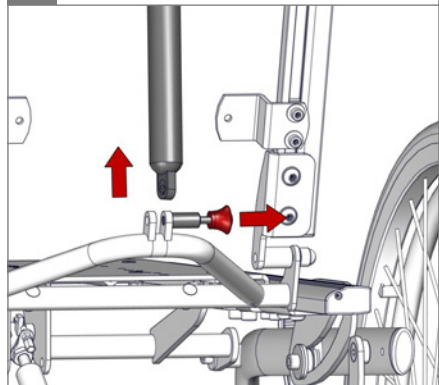
4.


5.

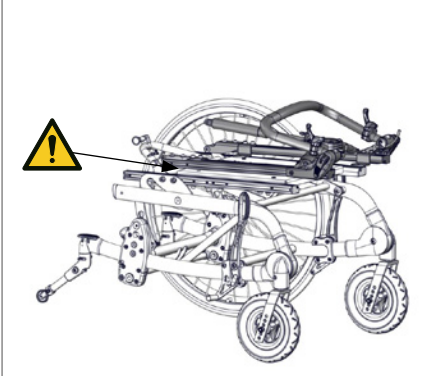01

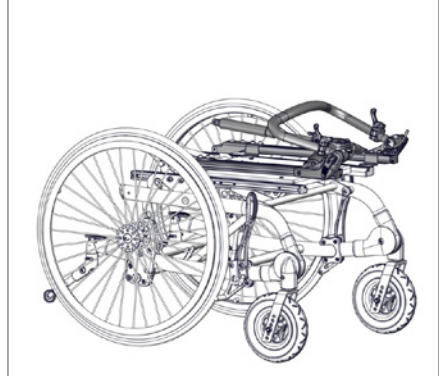6.

1.

01

2.

3.

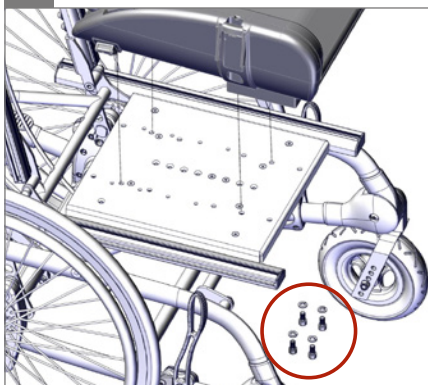
4.


5.

6 mm - 2,5 Nm

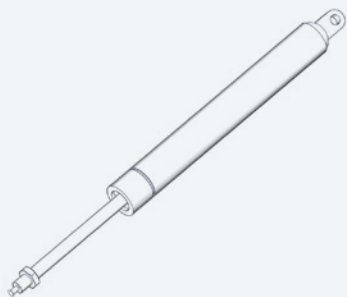
1.

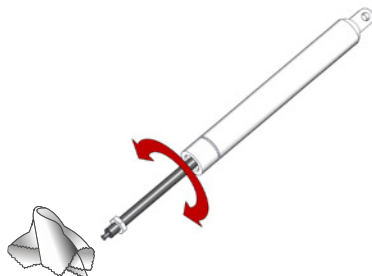2.






3.

 17 mm

4.

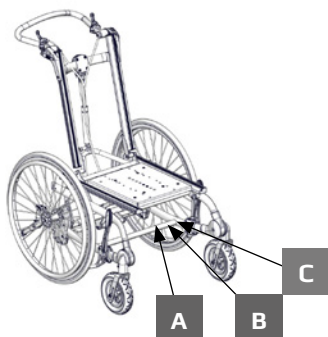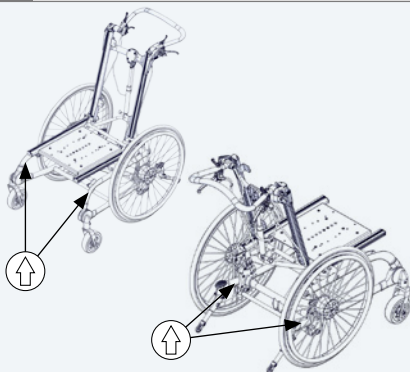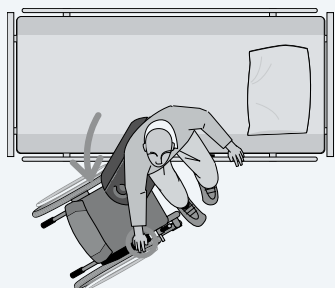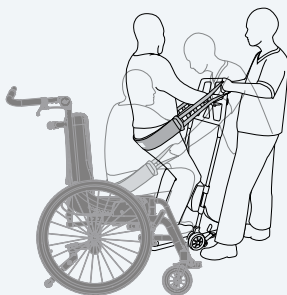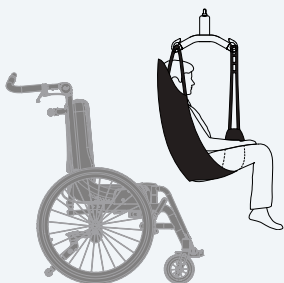 17 mm

A

XXXXXXX Max load: xxx kg
DMR ver.: XXXX Max load: xxx kg
Art. no.: XXXXXXXX
Product: XXXX - Size X

(xx)xxxxxxxxxxxxxxxxxxxxxx(00)xxxxxxxx(00)xxxxxxxx

B

R82
Parallelsvej 3
DK-8751 Gedved
R82.org
Made in XXXXXXXX


C


mm (inch)



	 1	 2	 3	 4
 Overall width	580 (22¾")	630 (24¾")	680 (26¾")	730 (28¾")
 Width between wheels	440 (17¼")	490 (19¼")	560 (22")	600 (23½")
 Length	830 (32¾")	830 (32¾")	830 (32¾")	830 (32¾")
 Overall height	900 (35½")	940 (37")	1040 (41")	1140 (45")
 Back height	430 (17")	470 (18½")	570 (22½")	670 (26½")
Folded height	550 (21¾")	550 (21¾")	550 (21¾")	550 (21¾")
Folded length	800 (31½")	800 (31½")	840 (33")	940 (37")
 Seat surface at front edge	430-460-490 (17-18-19¾")	430-460-490 (17-18-19¾")	430-460-490 (17-18-19¾")	430-460-490 (17-18-19¾")
Footrest to adaptorplate	100 - 250 (4 - 9¾")	100 - 250 (4 - 9¾")	180 - 330 (7 - 13")	180 - 330 (7 - 13")
 Seat plane angle	0-40° (5-45°)	0-40° (5-45°)	0-40° (5-45°)	0-40° (5-45°)
 Back angle	0 - 30°	0 - 30°	0 - 30°	0 - 30°
 Minimum turning radius	600 (23½")	600 (23½")	600 (23½")	600 (23½")
 Total mass	22 kg (48.5 lb)	23 kg (50.5 lb)	24 kg (53 lb)	27 kg (59.5 lb)
 Max. load	100 kg (220.4 lb)	100 kg (220.4 lb)	100 kg (220.4 lb)	100 kg (220.4 lb)
 Max. load at transportation	100 kg (220.4 lb)	100 kg (220.4 lb)	100 kg (220.4 lb)	100 kg (220.4 lb)
 Static stability downhill	9°	9°	9°	9°
 Static stability uphill	9°	9°	9°	9°
 Static stability sideways	9°	9°	9°	9°

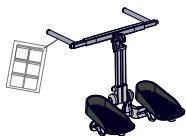9996097152


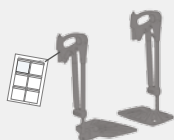
9996097153

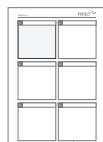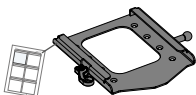
9996097714

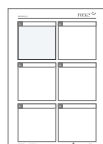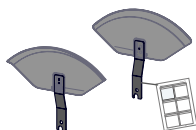
9996097134
9996097129
9996097128


9996097148
9996097150


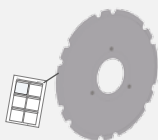
9996097137

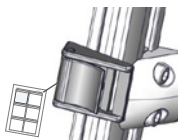9996097600

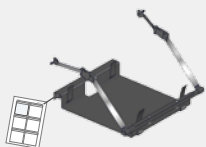9996097165

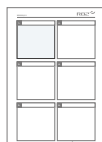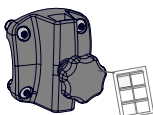9996097151

ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The Combi Frame:x is a wheelchair base for customized seating systems. Both users and caregivers benefit from the many built-in functions and simple adjustment possibilities. The Combi Frame:x is suitable for users needing an individual seating system and base, e.g. children with CP, GMFCS level 3-5. The Combi Frame:x comes in 4 sizes and is ideal for both indoor and outdoor use. For transportation the product is suitable for children in the weight range of 22 kg to specified max user weight for transportation.

Tools

There is a tool bag at the back of the chair with 3, 4, 5 and 6 mm Allen keys. 1 pcs. of T-tool is also included.

Declaration of Conformity -

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

- EN 12183

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the R82 homepage/download.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

ENGLISH

Symbols

For indoor and outdoor use

Consult instructions

The latest version of the instructions is always available on the R82 website and can be printed in larger sizes

Warning

This symbol appears in the User Guide along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger

01 = Entrapment Hazards

02 = Tilting Hazards

03 = Transport in motor vehicles

Prohibition

Incorrect lifting points, especially on stairs

Manual cleaning

Keep clean to maintain optimum function

Inspection

Listen to click sound

Correct lifting point

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

ENGLISH

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Always use the product in a moderate pace. Do not run with the product
- Information about transportation can be found in the document "Transportation in Motor Vehicles" supplied with the product
- Information regarding frame and seat combinations including max. load in combination can be found at: support.r82.org
- Information about roadability can be found in the document "Roadability" supplied with the product

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product
- Activate the anti-tips when the product is being used on inclines
- Activate the anti-tips before angling the seat and back
- Deactivate the anti-tips when forcing kerbs higher than 100mm
- Ensure the stability of the product is maintained before placing the user in it
- We recommend you notify the user before angling the seat and back
- Ensure the wheels are fully-functional and secure before use
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture.
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- The user must not occupy the product while it is being transported into or out of a vehicle by a lift

Environmental conditions

- Operating temperature: 5 °C to 40 °C
- Storage temperature: -10 °C to +50 °C
- Relative humidity: 20% to 90%
- Atmospheric pressure: 800 to 1060 hPa

ENGLISH

Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference
- Store the enclosed tools on/at the product

Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts
- Be aware of any MAX mark indications before adjustment

Weekly

- Check that all wheels are moving freely and that all wheel locks/brakes are easily activated
- Wipe off any residue and dirt from the product using a cloth wrung in warm water and a mild detergent/soap without chlorine and dry before use
- Disinfect the product. please check the disinfection section in this User Guide for more information

Monthly

- All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be standard or faulty

Washing machine

This product can be washed at 60° using a mild detergent in 15-20 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.

Disinfection

The product without accessories and cushion can be disinfected with a 70% disinfectant solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Service Check List:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- If a seat adapter is mounted, check that the red handle for locking works and that the safety lock "clicks into position when placing a seat into the adapter
- Wheels are moving freely and that all wheel locks/brakes are easily activated. Adjust if required
- Quick release wheels are removable/replaceable
- Tire pressure
- Spoke guards is without cracks and secured to the wheel with the clips mounted on each guard
- Before storage: Always deactivate wheel locks/brakes
- Do not place more than one user in the product
- Anti-tips can be fixed in working and resting position
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

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Supplier for:
USA, Canada

R82 UK Ltd.

Unit D4A,
Coombswood Business Park East
Coombswood Way,
Halesowen
West Midlands B62 8BH
England
Tel. +44 (0)121 561 2222
Fax +44 (0)121 559 5437
R82uk@R82.com
R82.uk
facebook.com/R82UK

Supplier for:
United Kingdom, Ireland

R82 GmbH

An den Wiesen 10
55218 Ingelheim
Germany
Tel +49 (0)6132 710 70
Fax +49 (0)6132 710 710
R82de@R82.com
R82.de
facebook.com/R82.de

Supplier for:
Germany, Switzerland, Austria

R82 Nederland B.V.

Tinweg 8
8445 PD Heerenveen
The Netherlands
Tel +31 (0)513 68 16 86
Fax +31 (0)513 68 19 84
R82nl@R82.com
R82.nl
R82.be

Supplier for:
The Netherlands, Belgium

Find your local dealer

www.R82.org

DISTRIBUTOR:

M1082

Reserving our rights for printer's error or discontinued products
Please find an updated edition on our website www.R82.org

© 2013 R82 A/S All rights reserved The R82 logo and the icons are registered trademarks of R82 A/S