690 mm (27¼")

<100 kg (220.4 lb)


Multi Frame User Guide


EN 12183

2018-07-rev.5

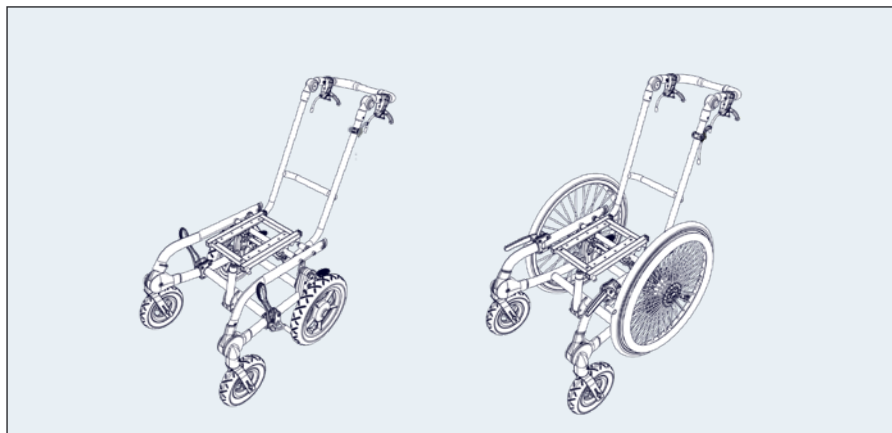
	3	ENGLISH DEUTSCH	36 - 38 39 - 42
	4 - 5	NEDERLANDS DANSK	43 - 46 47 - 49
	6	NORSK SVENSKA	50 - 52 53 - 55
	7 - 10	SUOMI ESPAÑOL	56 - 58 59 - 62
	11	FRANÇAIS ITALIANO	63 - 66 67 - 70
	12 - 13	PORTUGUÊS PORTUGUÊS DO BRASIL	71 - 74 75 - 78
	14 - 17	РОССИЯ POLSKI	79 - 82 83 - 85
	18	ČESKY 中國	86 - 88 89 - 91
	19 - 22	日本 ΕΛΛΗΝΙΚΗ	92 - 94 95 - 97
	23 - 24		
	25		
	26		
	27		
	28		
	28		
	29 - 32		
mm (inch)	33 - 34		
	35		


SZ 1

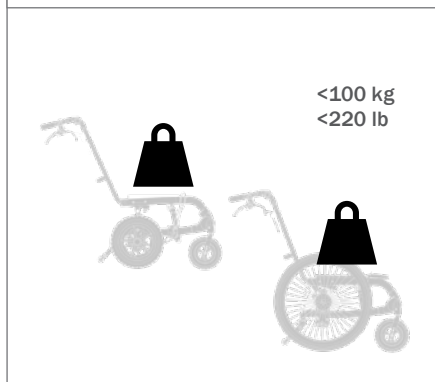
SZ 2

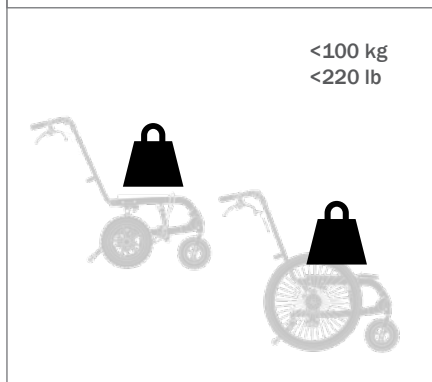
SZ 3

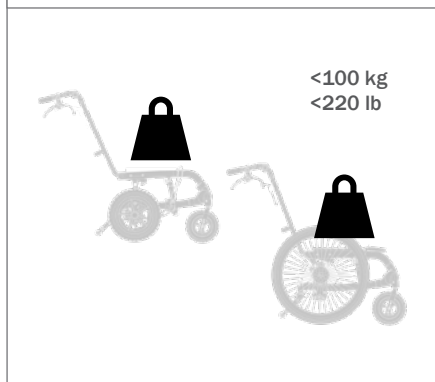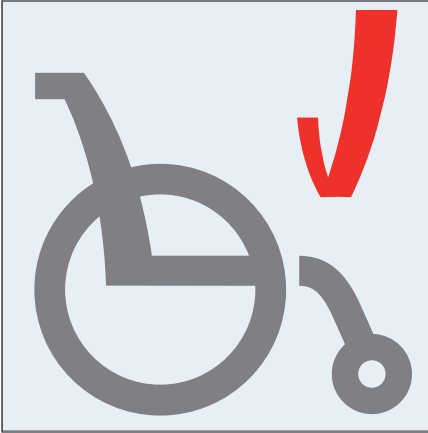
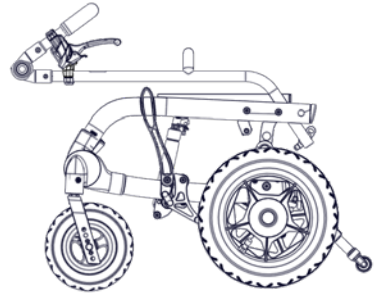
1.

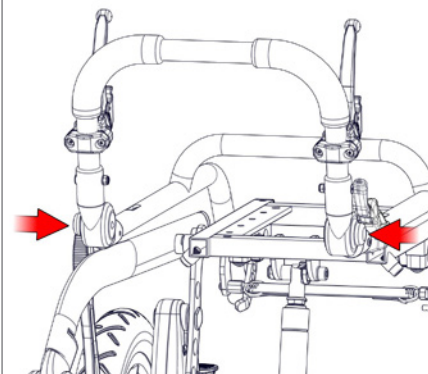
2.

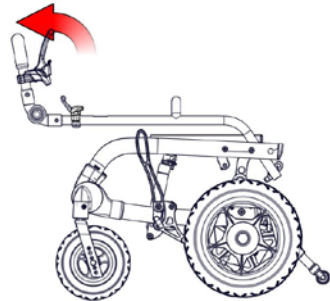
3.

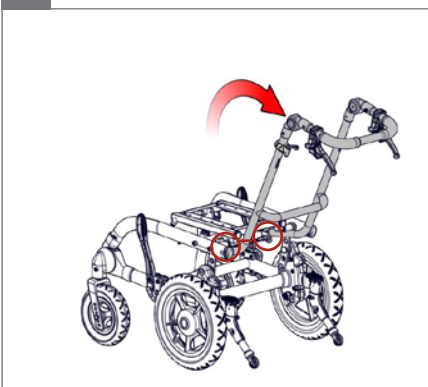4.

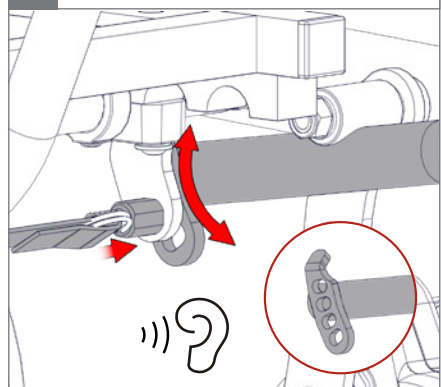
5.


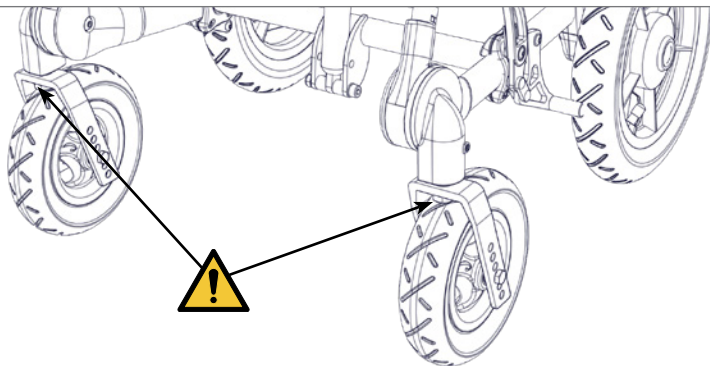
5.


 05

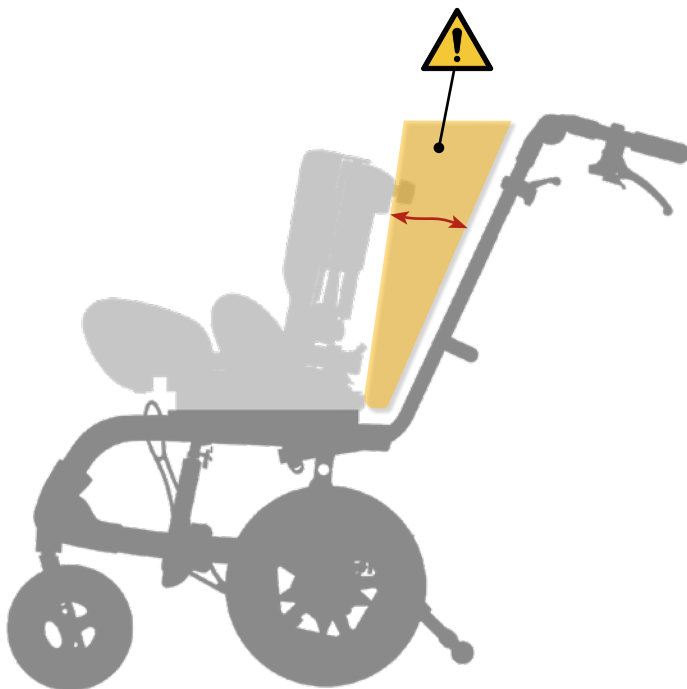6.

 05

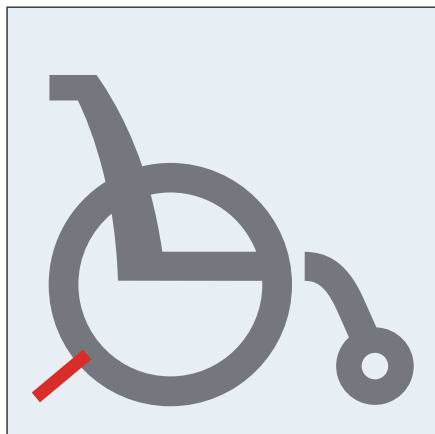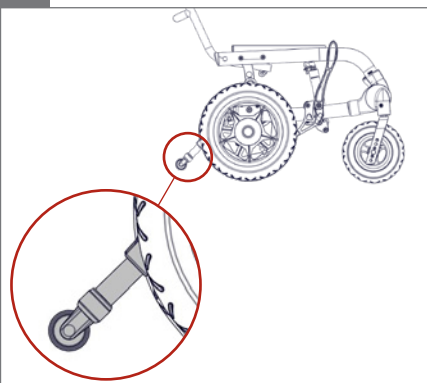
1.


2.

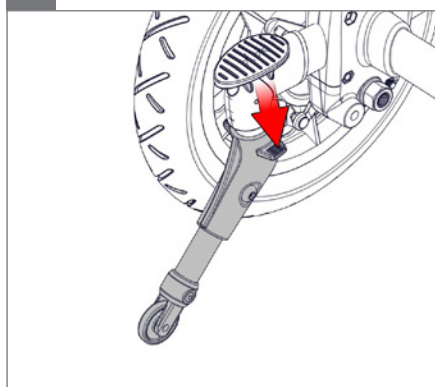
○ 3 mm

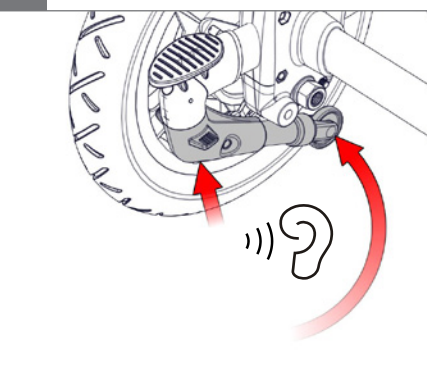
3.


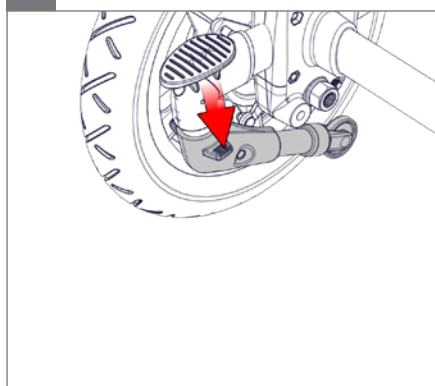
4.

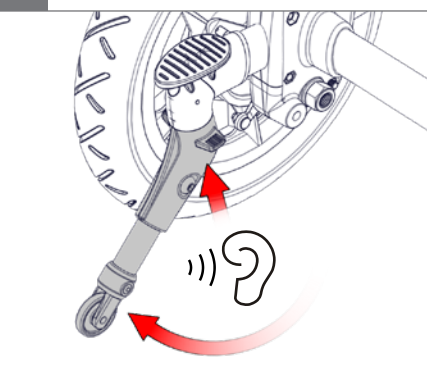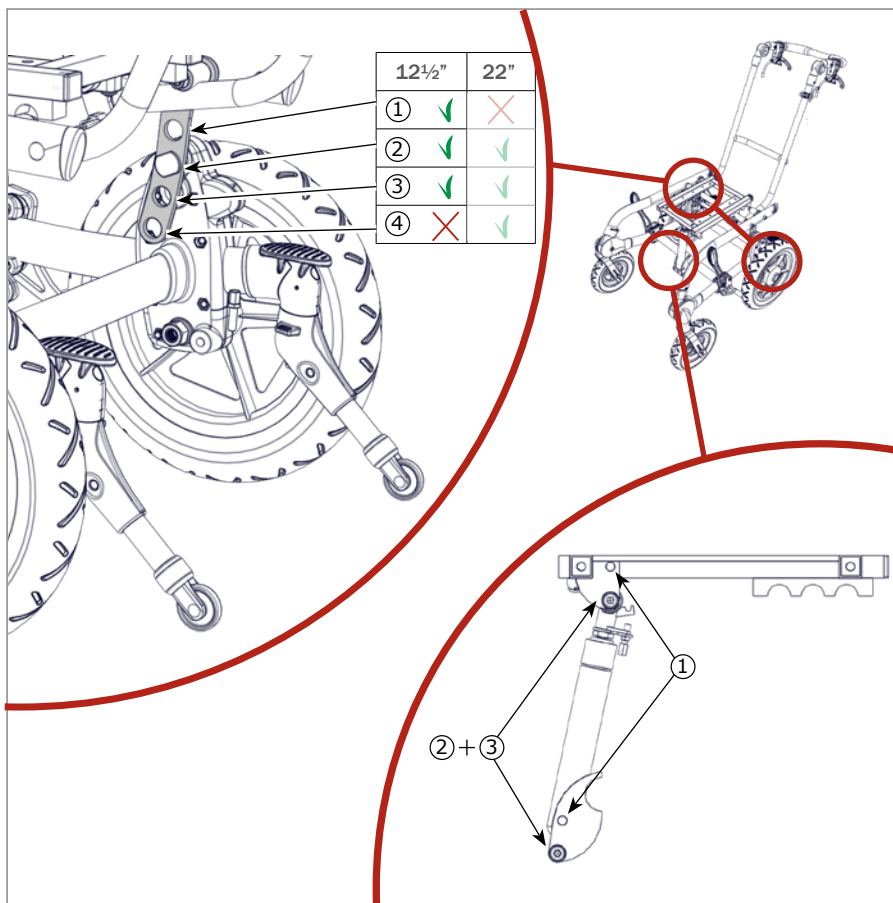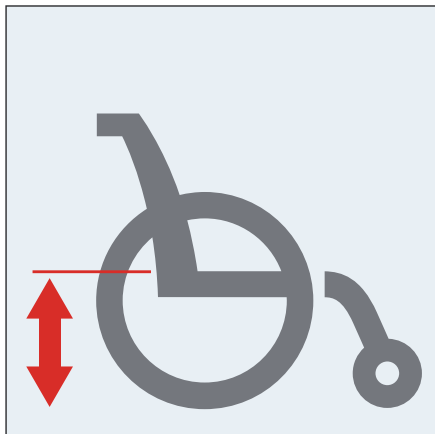5.



1a.

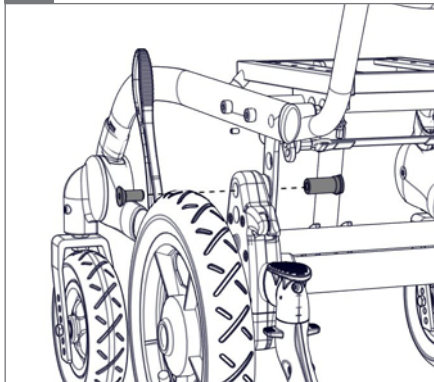⌀ 6 mm


1b.

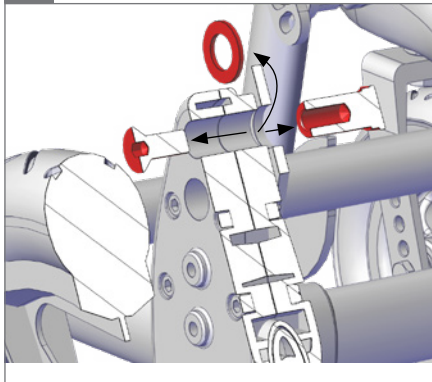
⌀ 6 mm

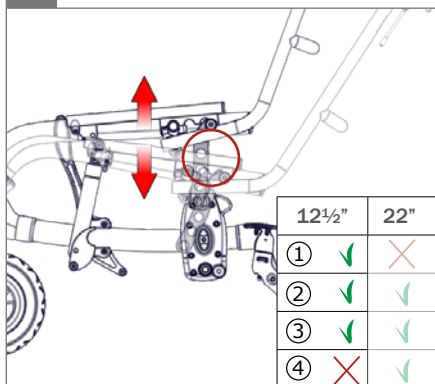
2.

3a.


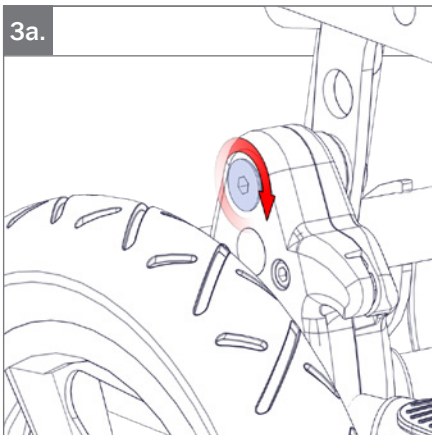
3b.

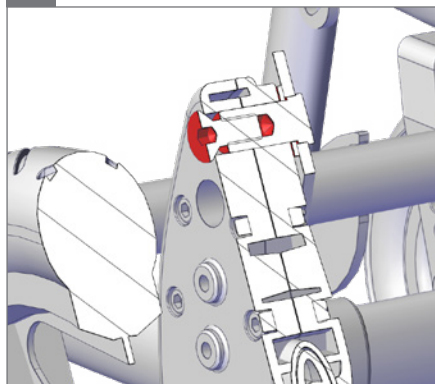4.

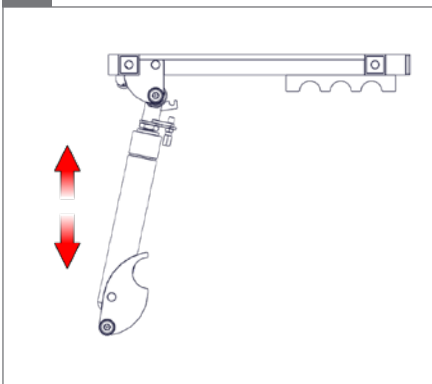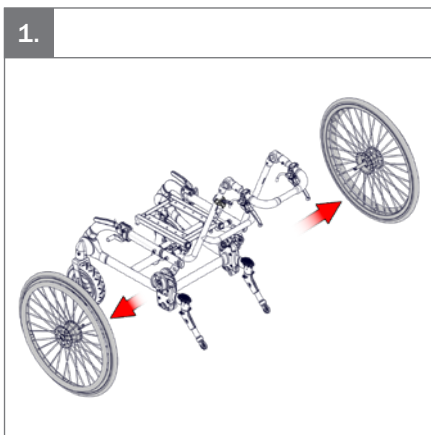
⌀ 6 mm - ⚓ 13 mm


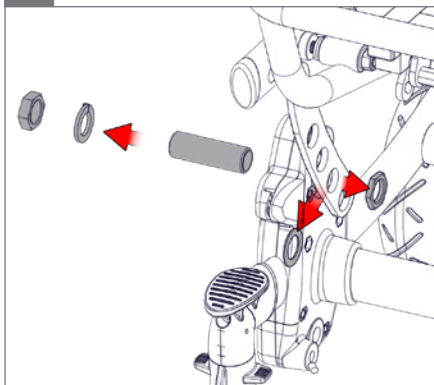
2.

2x  24 mm

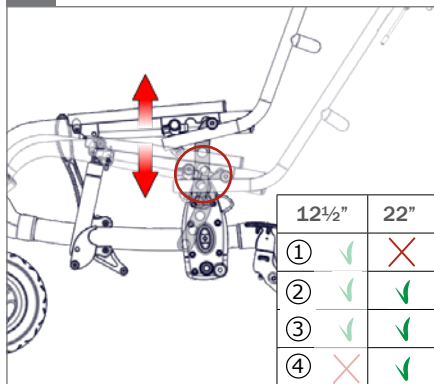
3.


4a.

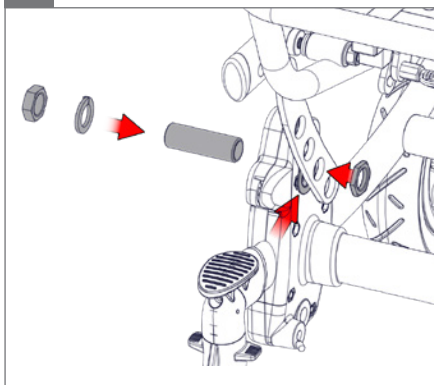2x  24 mm

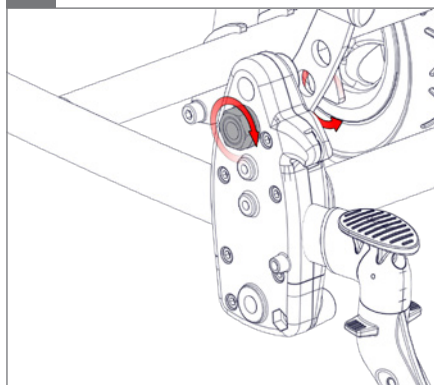
4b.

2x  24 mm

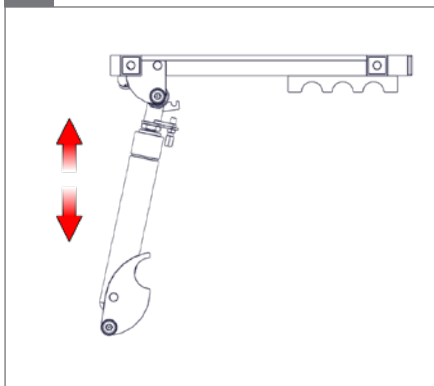5.


⬡ 6 mm -  13 mm

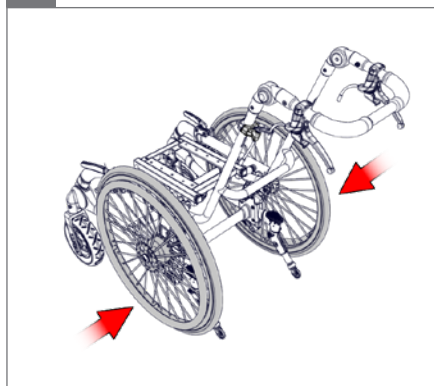6.


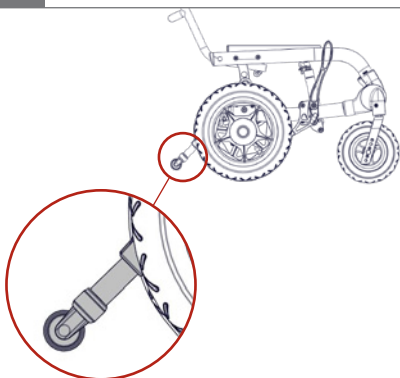
1.


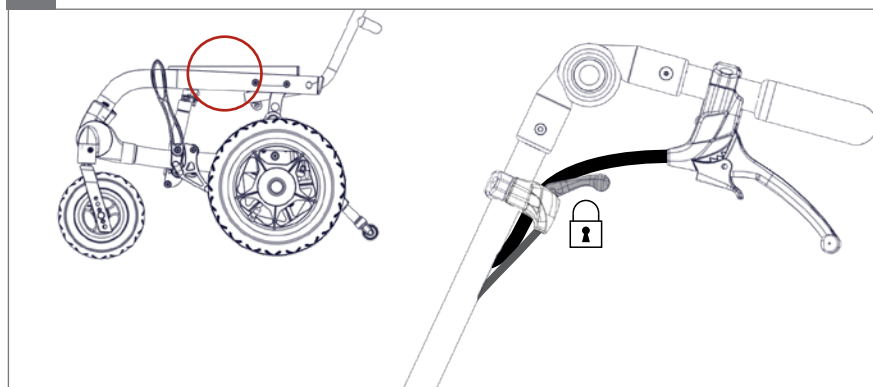
2.

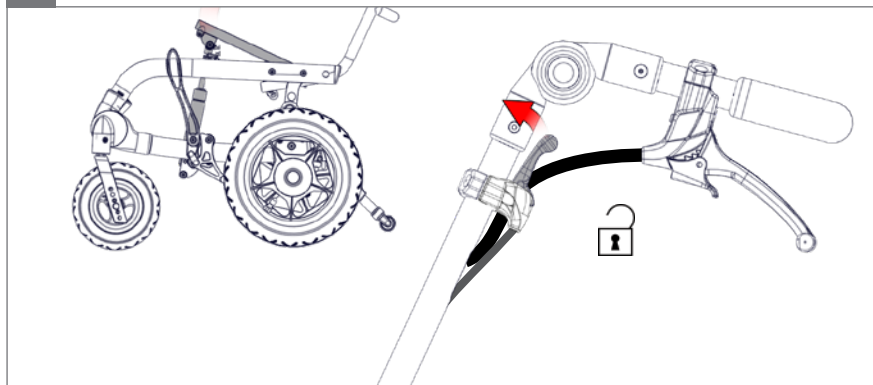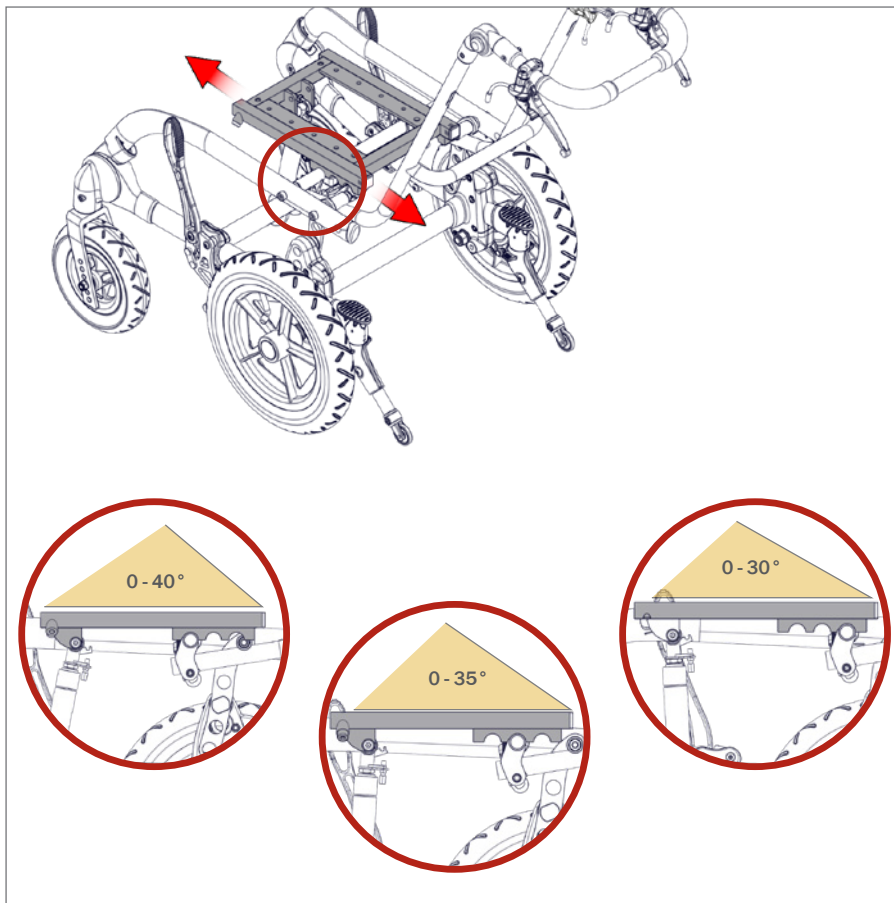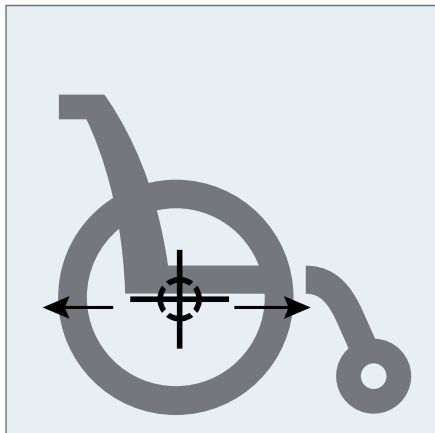3.



1.

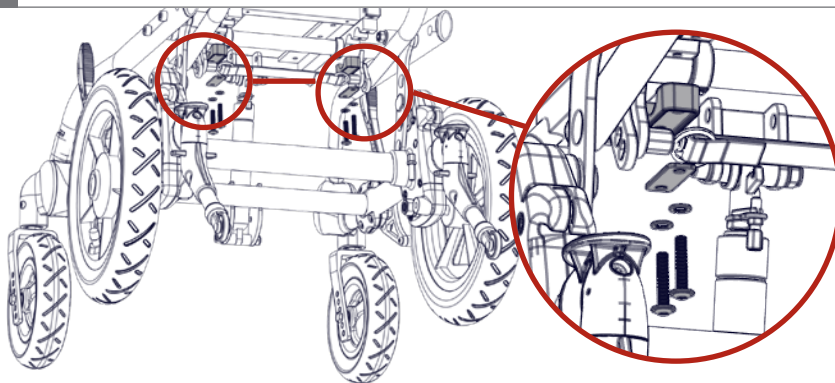
⌀ 4 mm

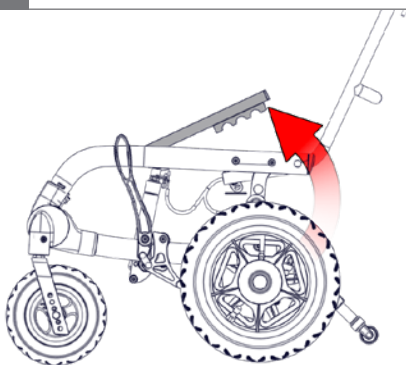
2.








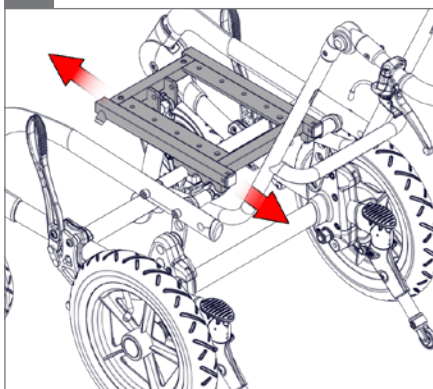
3.

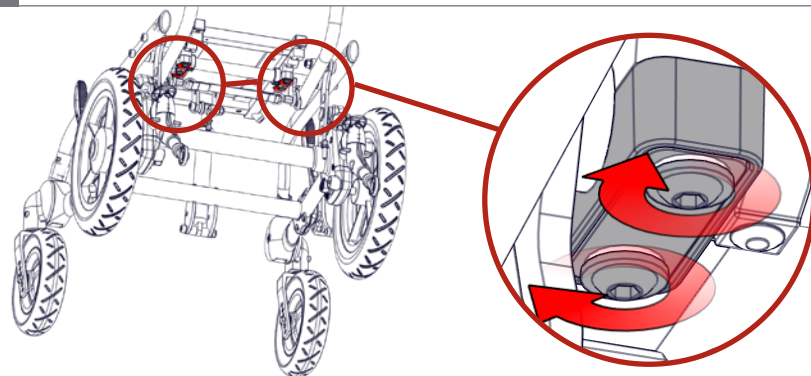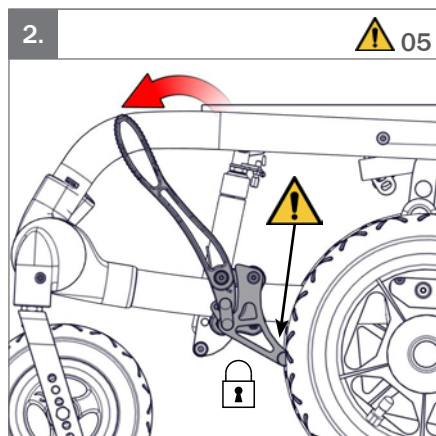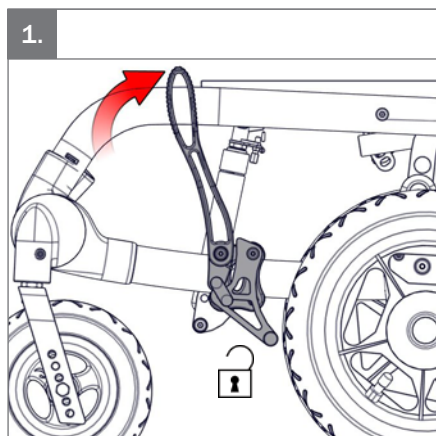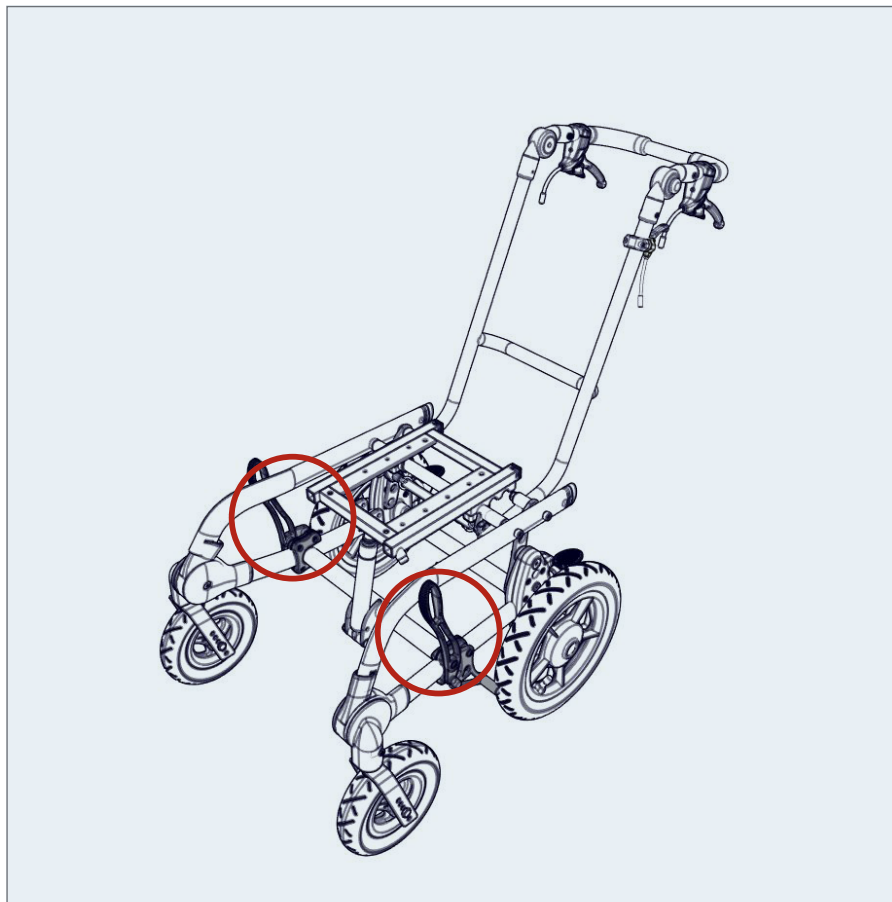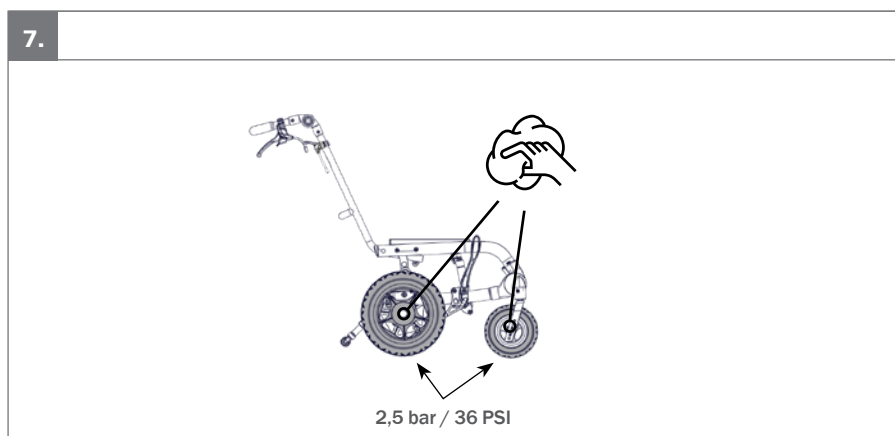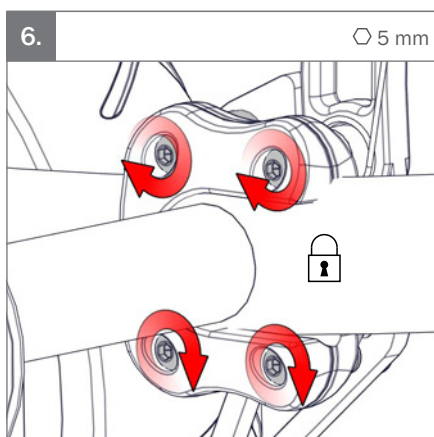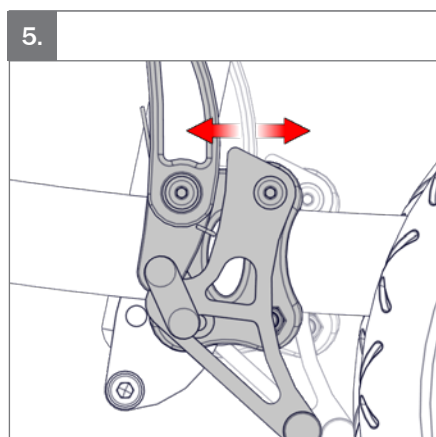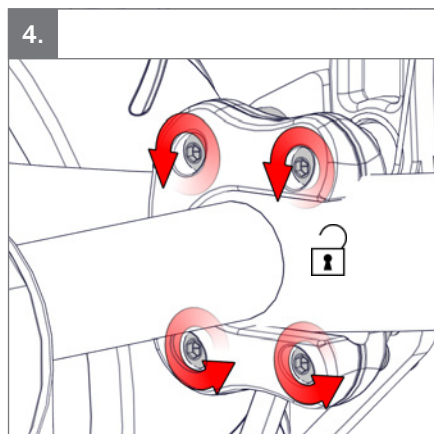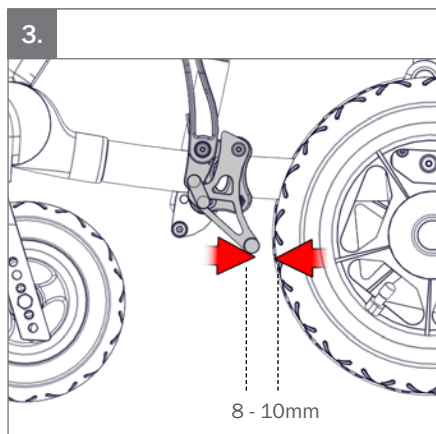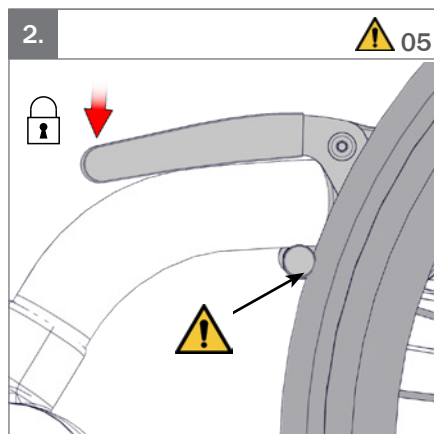
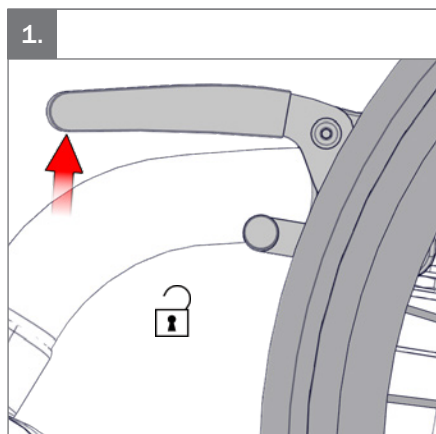
4.



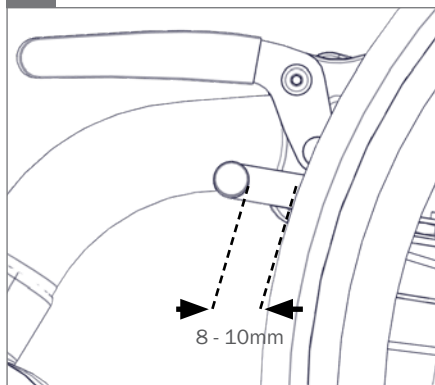
3.

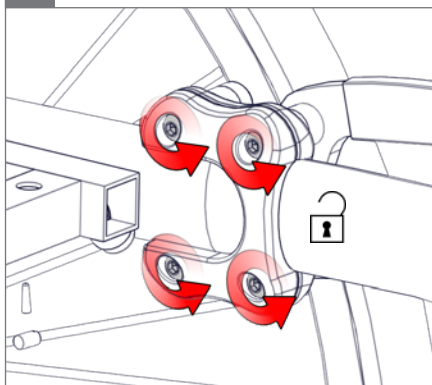4.


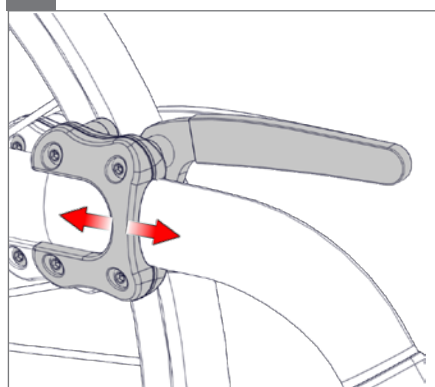
5.


6.

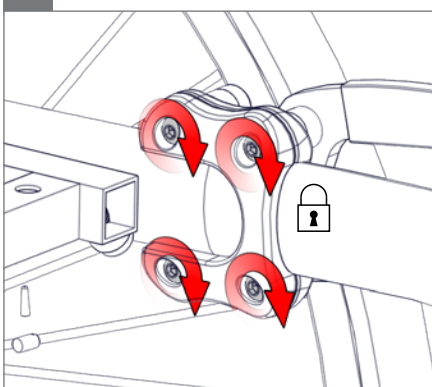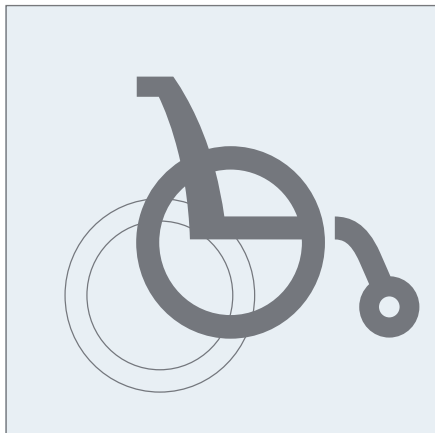○ 5 mm


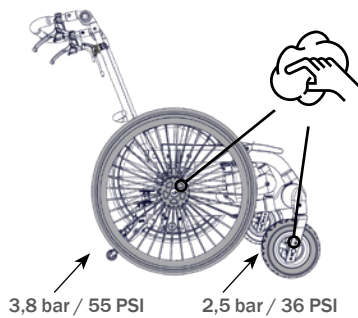
1.


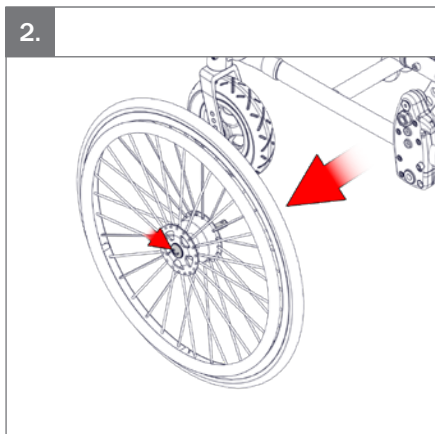
2.

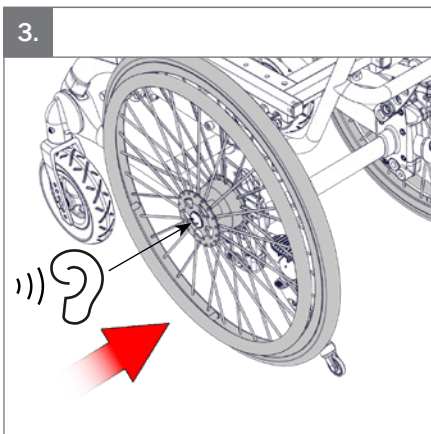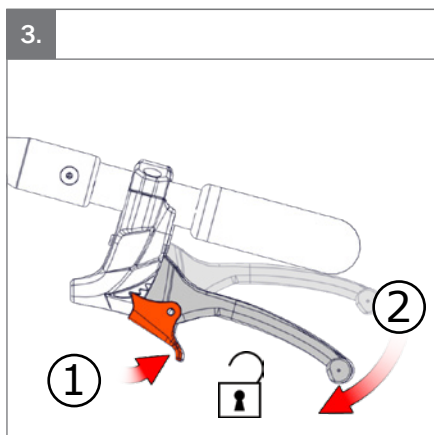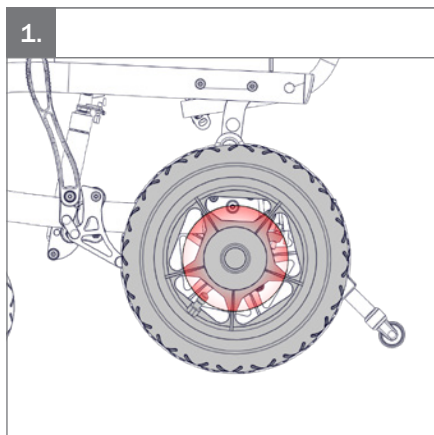
3.

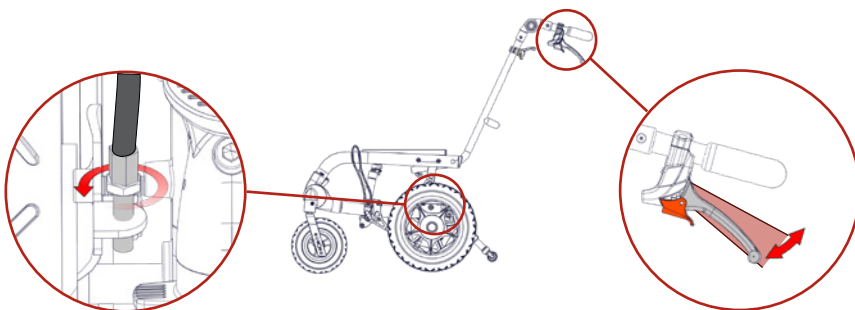
5.






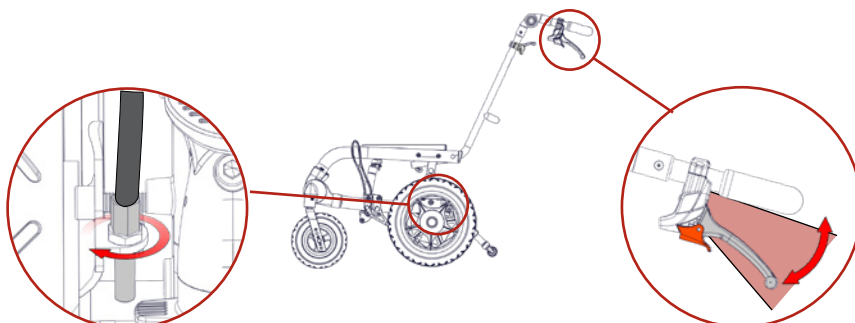
6.

4.

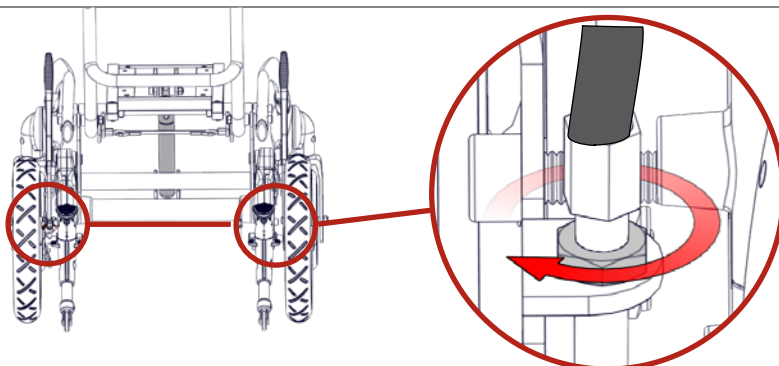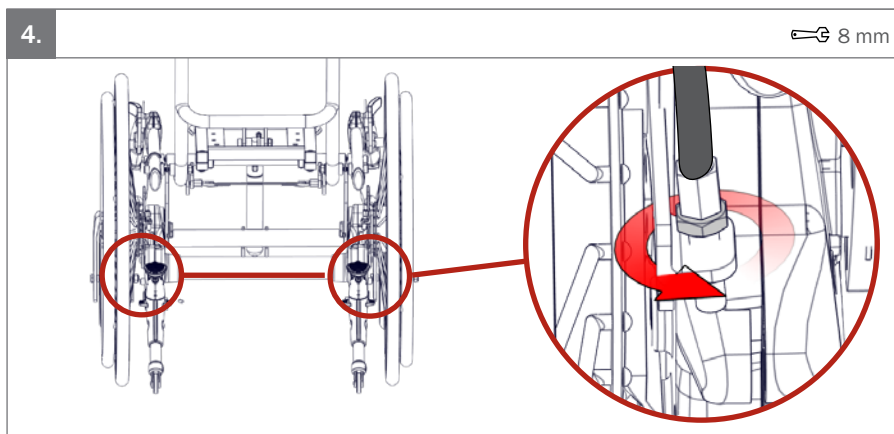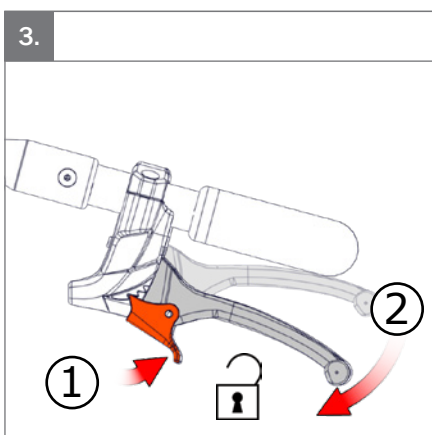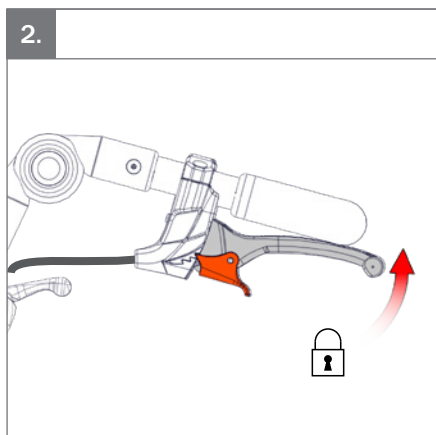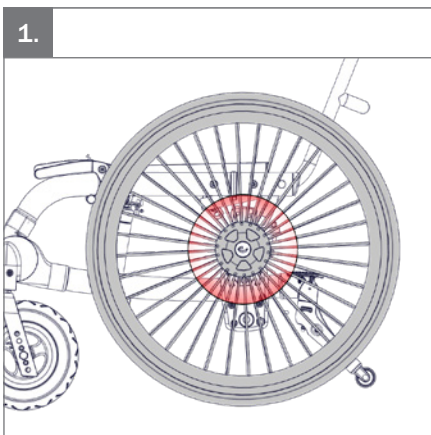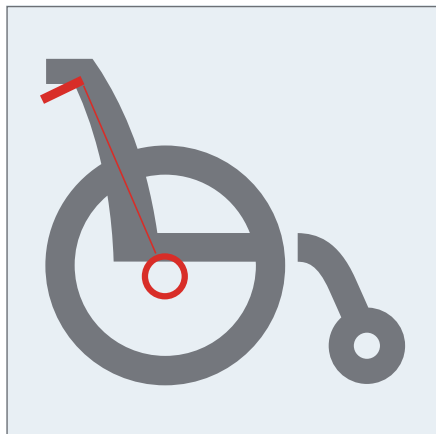
8 mm

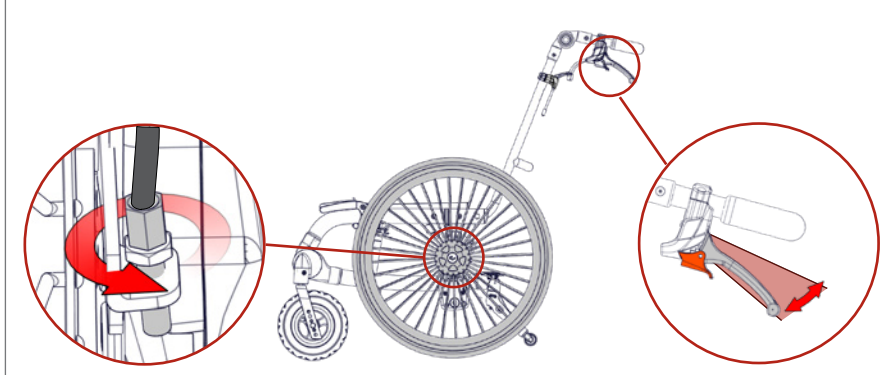
5.






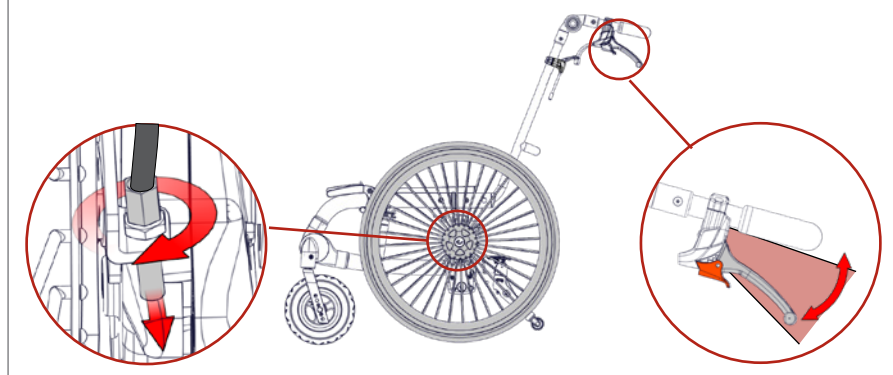
6.

7.

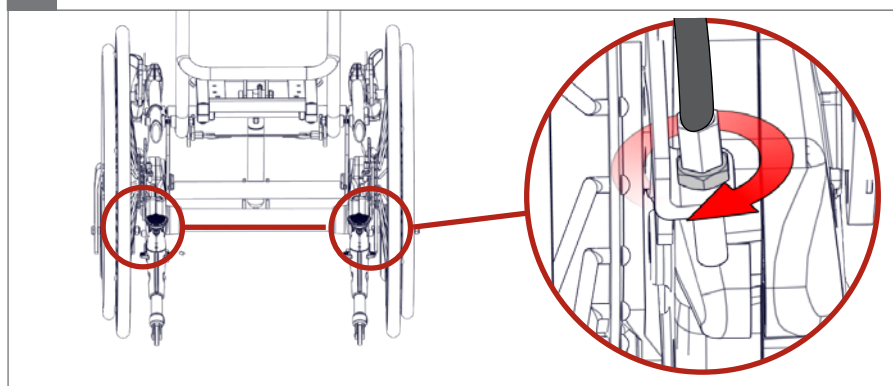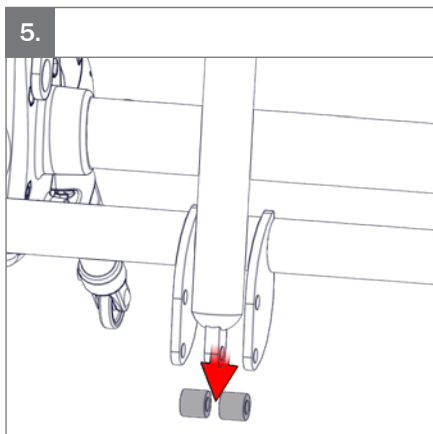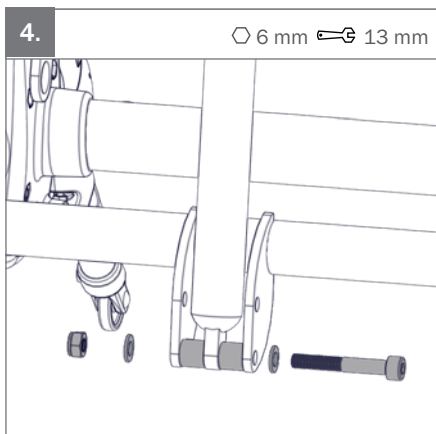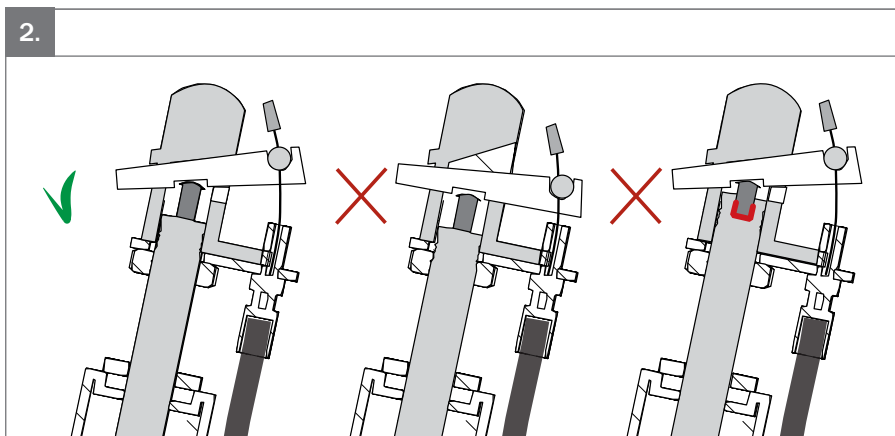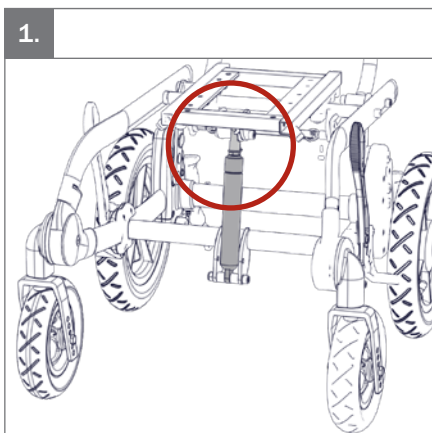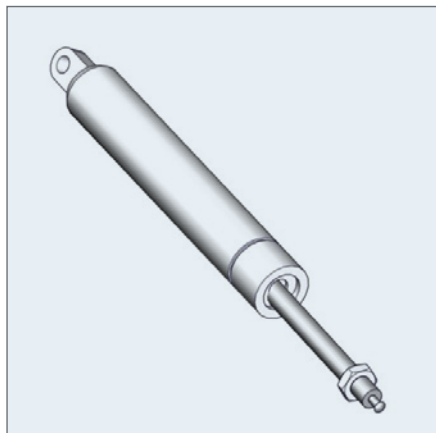8 mm



6.

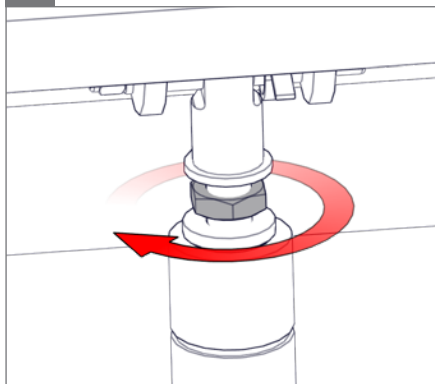
17 mm

7a.

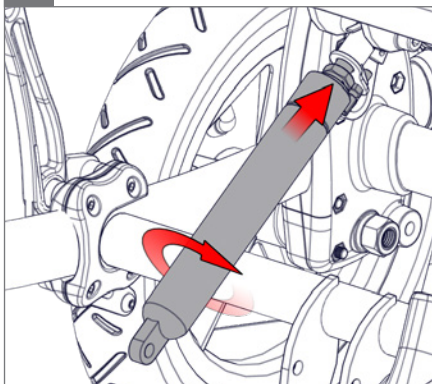
7b.

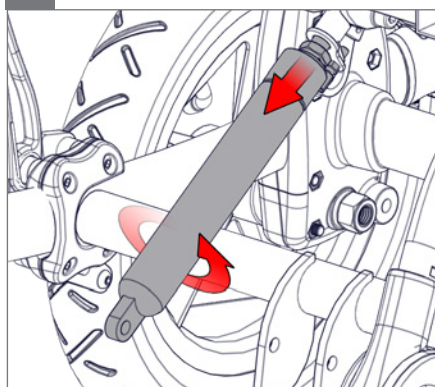
8.

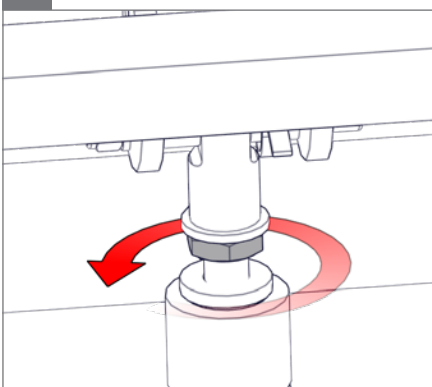17 mm


9.

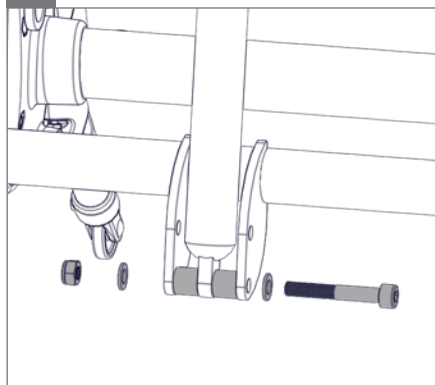6 mm 13 mm

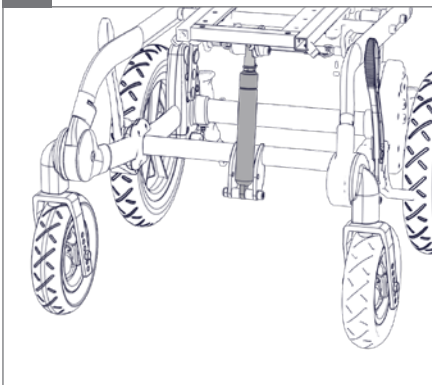
10.


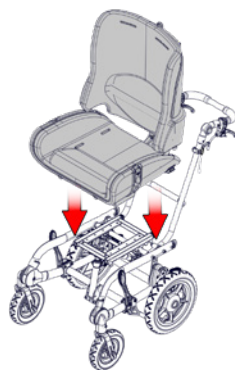
1. Panda Futura 5 & x:panda sz 3 & 4

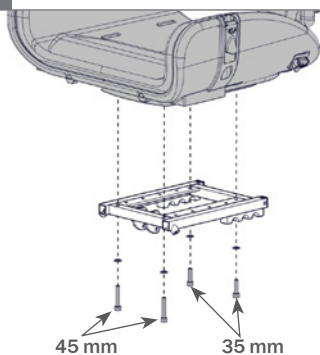2.



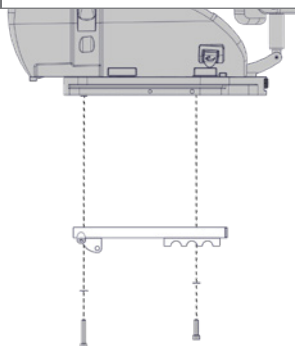
3.


4.

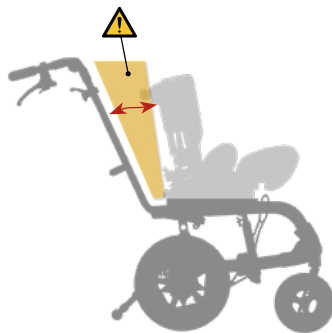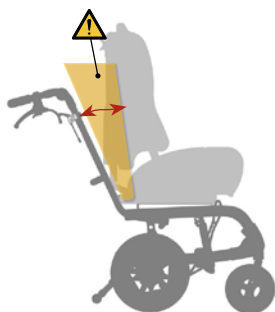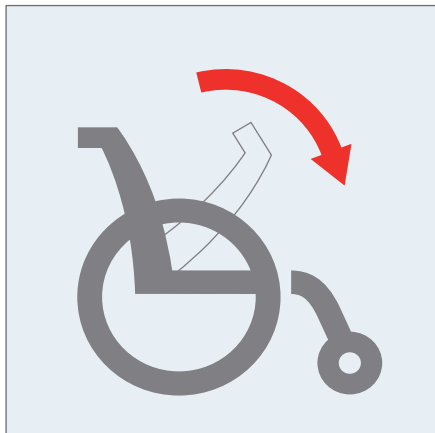 05


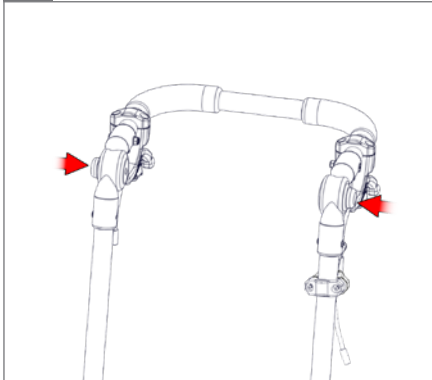
1.

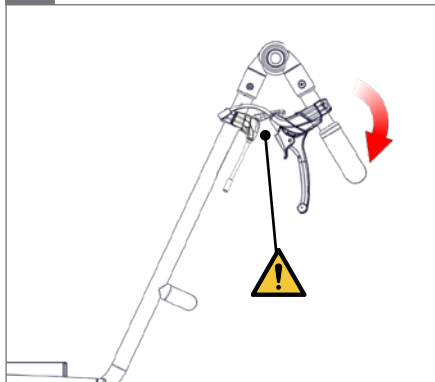
2.



 05



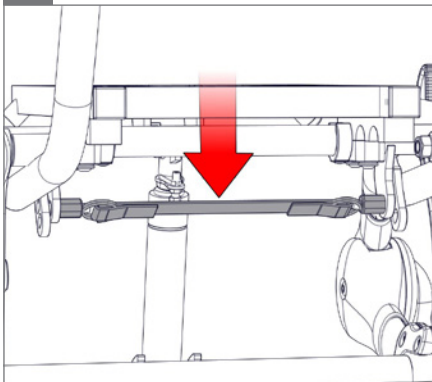
3.

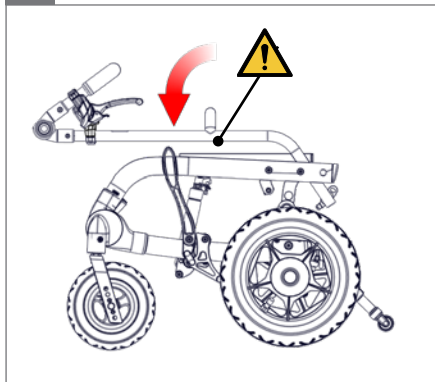4.

 05

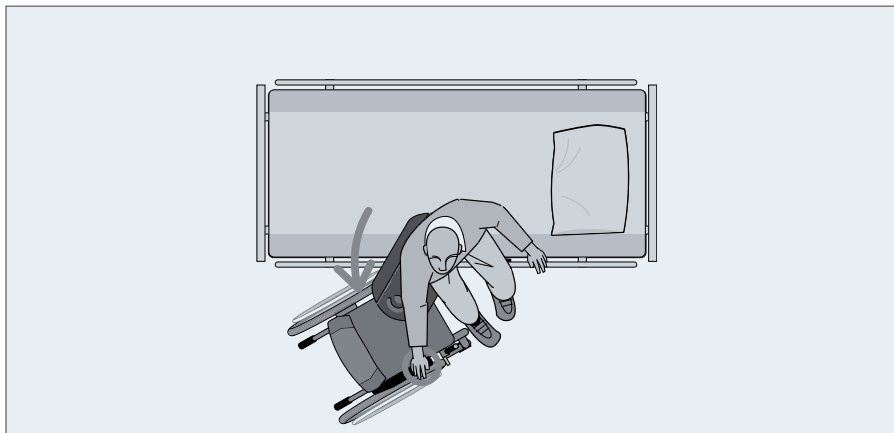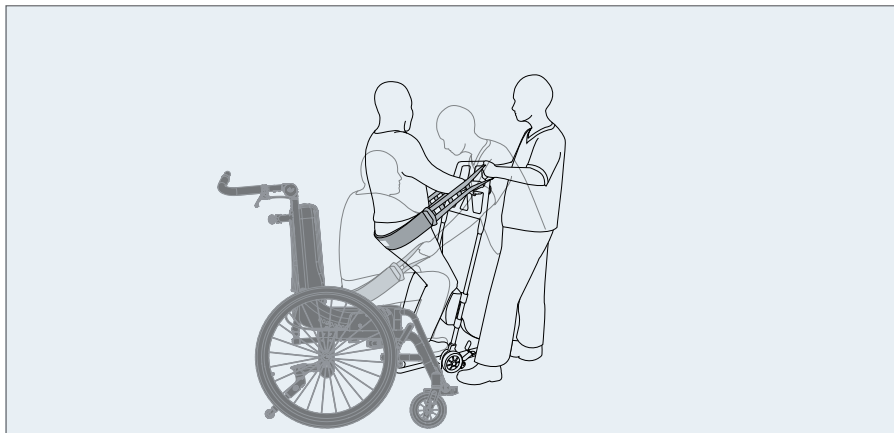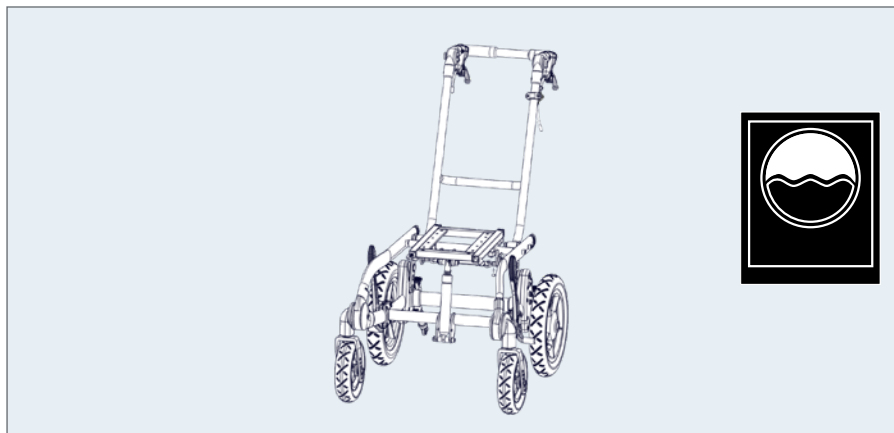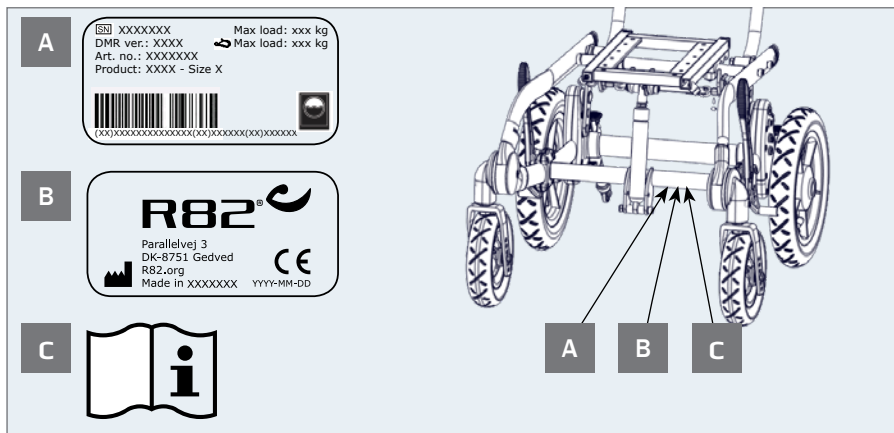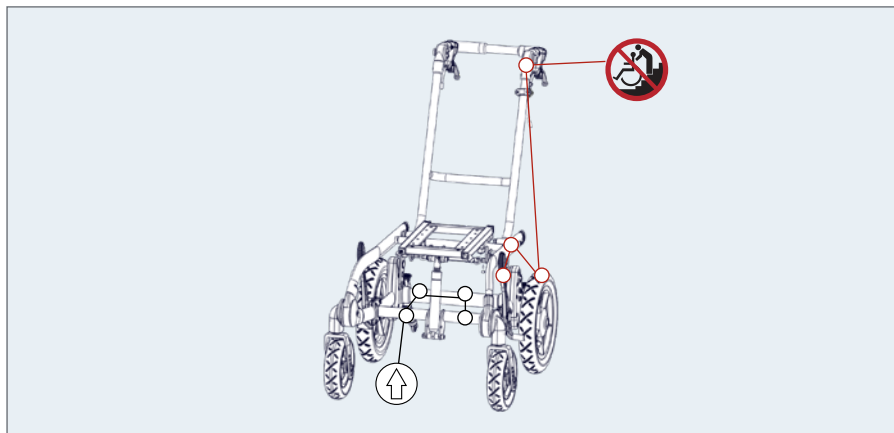EN: For indoor and outdoor use

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

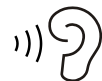
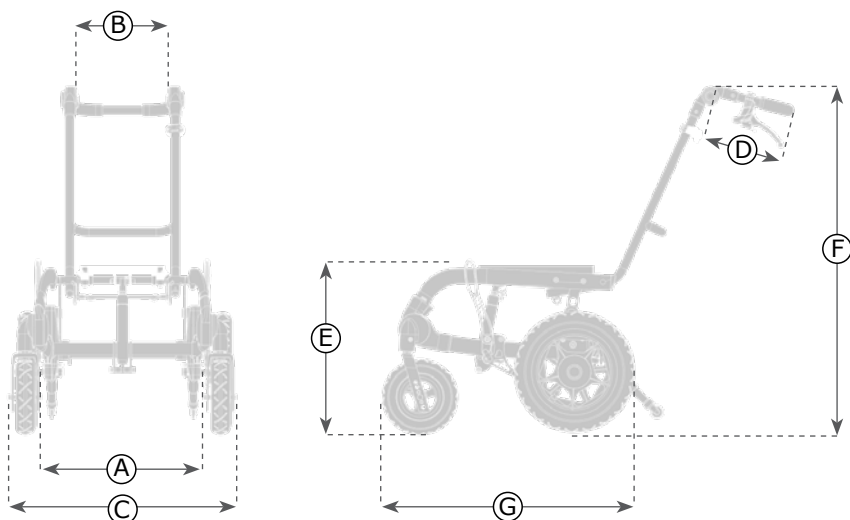
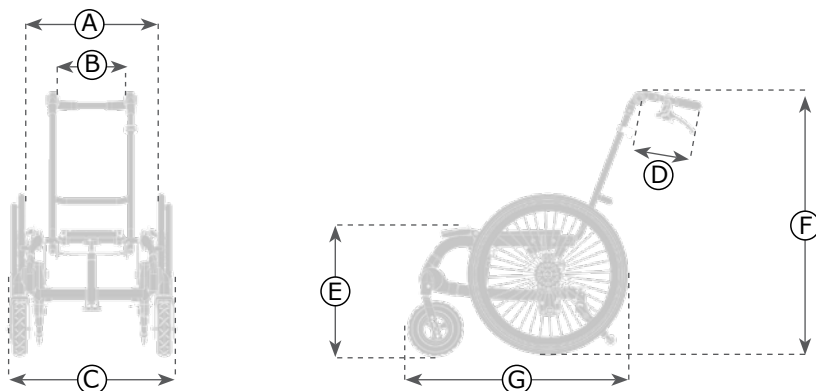
mm (inch)



	1	2	3
 Width, inside (B)	340 mm (13½")	390 mm (15¼")	440 mm (17¼")
 Height floor-basic plate (E) (top/mid/bottom position)	395 / 425/ 455 mm (15½ / 16¾ / 18")	395 / 425/ 455 mm (15½ / 16¾ / 18")	395 / 425/ 455 mm (15½ / 16¾ / 18")
 Back width, inside (B)	245 mm (9¾")	295 mm (11½")	345 mm (13½")
 Overall width 22" (C)	590 mm (23¾")	640 mm (25¼")	690 mm (27¼")
 Distance between 22" wheels w/ drum brakes (A)	430 mm (17")	480 mm (19")	530 mm (20¾")
 Distance between 22" wheels w/ drum brakes (A)	440 mm (17¼")	490 mm (19¼")	540 mm (21¼")
 Length, with 22" wheels (G)	750 mm (29½")	750 mm (29½")	800 mm (31½")
 Overall height push brace 15° (F)	915 mm (36")	915 mm (36")	915 mm (36")
 Overall height push brace 30° (F)	850 mm (33½")	850 mm (33½")	850 mm (33½")
Hand brace length (D)	225 mm (8¾")	225 mm (8¾")	225 mm (8¾")
 Seat plane angle	0-30° / 35° / 40°	0-30° / 35° / 40°	0-30° / 35° / 40°
 Folded height, without back wheels (mid position)	400 mm (15¾")	400 mm (15¾")	400 mm (15¾")
 Folded length, with 22"	850 mm (33½")	850 mm (33½")	850 mm (33½")
 Handrim diameter	480 mm (19")	480 mm (19")	480 mm (19")
 Footrest to adaptorplate	50 - 340 mm (2-13½")	50 - 340 mm (2-13½")	50 - 340 mm (2-13½")
 Max load	100 kg (220.4 lb)	100 kg (220.4 lb)	100 kg (220.4 lb)
 Max load at transportation	100 kg (220.4 lb)	100 kg (220.4 lb)	100 kg (220.4 lb)
Total mass (22" air wheels+air front wheels)	14 kg (30,8 lb)	14 kg (30,8 lb)	14 kg (30,8 lb)

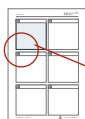
9996097118


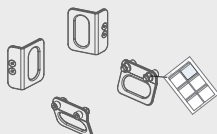
9996097135

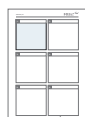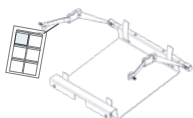9996097165

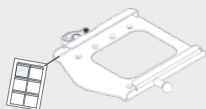
9996097714

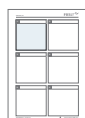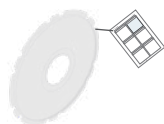9996097737

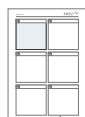9996097738

9996097755

ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

Multi Frame is a smart and simple wheelchair frame designed for children and teenagers. The product is suitable for users needing an individual seating system and base for indoor and outdoor use.

The Multi Frame is suitable for users needing an individual seating system and base, e.g. Children/youngsters with CP at GMFCS level III-V, Strong extensor spasticity and Developmental delays (physical or mentally handicapped).

Configurations

The product is available in multiple configurations. Components and functionalities vary between the configurations. Disregard any component or functionality explained in this User Guide not part of your particular configuration.

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

- EN 12183

Enclosed tools:

- 4, 5 and 6 mm Allen key
- 10 and 13 mm spanner

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the R82 homepage/download.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

Presales information

- support.r82.org

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- Be aware of any MAX mark indications before adjustment
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Always use the product in a moderate pace. Do not run with the product
- Information about transportation can be found in the document "Transportation in Motor Vehicles" supplied with the product
- Information about roadability can be found in the document "Roadability" supplied with the product

Environment

- Observe the surface temperature on the product, before placing a user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered
- For transportation the product is suitable for children in the weight range of 22 kg to specified max user weight for transportation

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Ensure the stability of the product is maintained before placing the user in it
- Activate the brakes before placing the user into the product
- Activate the anti-tips before angling the seat and back
- Deactivate the anti-tips when forcing kerbs higher than 100 mm
- We recommend you notify the user before angling the seat and back
- Ensure the wheels are fully-functional and secure before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Inspect the product and all of its accessories and replace any worn out parts before use
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1, EN 1021-2 and ISO 7176-16
- The product might lose its flame resistant characteristics if using other than original R82: Cushions

Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference
- Store the enclosed tools on/at the product

Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts
- Be aware of any MAX mark indications before adjustment

Weekly

- Check that all wheels are moving freely and that all wheel locks/brakes are easily activated
- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use
- Disinfect the product. please check the disinfection section in this User Guide for more information
- Tire pressure

Monthly

- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

WHEELS

Wheel axles: The axles should be kept free of dirt. Clean as frequently as necessary.

Ball bearings: The ball bearings require no maintenance.

Handrims: If a handrim should be damaged in such a way that it could lead to hand injury, it should be replaced.

22" tires: Check the air pressure (recommended: 55 PSI/3,8 bar/380 kPa) and tread at a regular basis. For instance by your local bicycle engineer.

12½" tires: Check the air pressure (recommended: 35 PSI/2,5 bar/240 kPa) and tread at a regular basis. For instance by your local bicycle engineer.

Brakes

The function of the brakes is dependent upon tire pressure. Caked on dirt can have a negative effect on the brake mechanism. The brakes should be checked at least once a month.

For safety routinely check to make sure the distance between the wheel and the wheel lock block is: 8-10 mm. The brakes should not be activated when the frame is stored as this could result in a flat spot in the solid inner of the tyre. If drum brakes are mounted, the cables have to be adjusted, to ensure the brakes are working properly. Make sure the wheels can not be turned around when the brakes are activated.

Warewashing machine

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

Disinfection

The product without accessories and cushion can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Service Check List:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- If a seat adapter is mounted, check that the red handle for locking works and that the safety lock "clicks into position when placing a seat into the adapter
- Wheels are moving freely and that all wheel locks/brakes are easily activated. Adjust if required
- Quick release wheels are removable/replaceable
- Anti-tips can be fixed in working and resting position
- Tire pressure
- Spoke guards is without cracks and secured to the wheel with the clips mounted on each guard
- The Pushbrace can be folded over the frame
- Before storage: Always deactivate wheel locks/brakes
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

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R82 Nederland B.V.
R82 Inc.
R82 au Pty Limited