

**Flamingo (seat)**

User information appropriate for use by people with visual, reading or cognitive disability      The latest version of all instructions is always available on the R82 website [support.R82.org](http://support.R82.org) and can be printed in larger sizes

Information available in pictogram      The instructions are made as pictograms

Intended use and intended environment including standard options      The Flamingo seat is designed to make the toilet-/bathing situation as pleasant and comfortable for both the user and caregiver. Use the Flamingo seat mounted on a standard toilet, as a bathing seat in a tub or on one of the available frames. The seat comes in four sizes and is suitable for users with GMFCS level 3-5. The Flamingo seat is only for indoor use but can in an emergency be taken outside.

Maintenance instructions      Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

Frequency	Description
1 <sup>st</sup> day	- Read the User Guide carefully before use and store it in a safe place for future reference
Before every use	- Disinfect the upholstery. please check the disinfection section in this User Guide for more information
Daily	- Use a cloth with warm water and a mild detergent to clean the product - Visual inspection. Repair or replace any damaged or worn-out parts - Visual inspection. Repair or replace any damaged or worn-out parts - Check all fittings and buckles for signs of worn out parts - Check that all castors are moving freely and that all wheel locks/brakes are easily activated - Ensure that all fasteners are in place and fitted correctly - Be aware of any MAX mark indications before adjustment
Weekly	- Disinfect the product. please check the disinfection section in this User Guide for more information
Monthly	- Nuts and bolts on the product are tightened - Oil the swing-away parts
Annually	- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

Cleaning description      This product can be washed at 85° using a mild detergent for 3 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

Flamingo (seat)	
Disinfection description	The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.
Dimensions (Incl. max load)	<a href="http://www.R82.org">http://www.R82.org</a> - Select product or click <a href="#">here</a>
Mass	N/A
Used in combination with other products	N/A
Warning about dangerous combinations	The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings.
List of accessories, detachable parts and materials	<a href="http://www.R82.org">http://www.R82.org</a> - Select product or click <a href="#">here</a>
Programmable controller information	N/A
Operator control adjustments	Manual
Folding and dismantling	N/A
Transport instructions (unoccupied)	Land and/or air
Sound power level	N/A
Turning diameter	NA
Forces on the wall	N/A
Electromagnetic environment warning	N/A
Electromagnetic influence warning	N/A
Life-time	The lifetime of the product in normal use is 5 years