

Toucan

User information appropriate for use by people with visual, reading or cognitive disability The latest version of all instructions are always available on the R82 website support.R82.org and can be printed in larger sizes

Information available in pictogram The instructions are made as pictograms

Intended use and intended environment The Toucan is a modern standing frame providing users the possibility of standing and having weight bearing through their feet. It is important to have the possibility to stand at an early age. Further it is not about standing for a long time but about standing often. The Toucan is suitable for many users e.g. children with CP, GMFCS level 3-4.

The Toucan comes in 5 sizes and is only for indoor use.

Maintenance instructions Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

Frequency	Description
1 st Day	<ul style="list-style-type: none"> - Read the User Guide carefully before use and store it in a safe place for future reference - Store the enclosed tools on/at the product
Daily	<ul style="list-style-type: none"> - Use a dry cloth to clean the product - Visual inspection. Repair or replace any damaged or worn-out parts - Ensure that all fasteners are in place and fitted correctly - Check all fittings and buckles for signs of worn out parts - Be aware of any MAX mark indications before adjustment
Weekly	<ul style="list-style-type: none"> - Check that all castors are moving freely and that all wheel locks/brakes are easily activated - Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use - Check that the angle mechanism functions correctly and that the safety lock is working correctly
Monthly	<ul style="list-style-type: none"> - Disinfect the product. - All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures - Oil the swing-away parts. We recommend you use a professional lubricant system -

Toucan	
	Annually <ul style="list-style-type: none"> - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty - The product has to be checked and tightened by a qualified technician at least every 12 months to avoid failures.
Cleaning description	This product can be washed at 60°C using a mild detergent in 15-20 minutes in a washing machine de-signed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.
Disinfection description	The product can be disinfected with a 70% disinfectant solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.
Dimensions	http://www.R82.org - Select product or click here
Mass	http://www.R82.org - Select product or click here
Used in combination with other products	N/A
Warning about dangerous combinations	The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings.
List of accessories, detachable parts and materials	http://www.R82.org - Select product or click here
Programmable controller information	N/A
Operator control adjustments	N/A
Folding and dismantling	N/A
Transport instructions	N/A
Sound power level	N/A